J3 Carbon TM Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a J3 Carbon Back is appropriate.

⚠ WARNING!

Backs should only be installed by:

- An authorized Sunrise Medical supplier.
- A qualified and competent healthcare professional who has undergone appropriate certified training by either sunrise medical or their approved supplier.

THE JAY J3 CARBON BACK INTENDED USE

The Jay J3 Carbon Back is intended to provide a back support for a wide variety of wheelchair users. It is designed to provide simple, comfortable and effective support to help maximise function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

The back can be installed onto a wide range of wheelchairs.

Maximum user weight: 125 kg (275 lbs).

The intended lifetime of this product is 5 years.

⚠ WARNING!

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

⚠ WARNING!

Do not use J3 Carbon Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

⚠ WARNING!

Do not use the J3 Carbon back handle to lift wheelchair. It is intended only for back removal.


⚠ WARNING!

Carelessness causes fire.

⚠ WARNING!

Please read this manual before usage.

J3 CARBON BACK

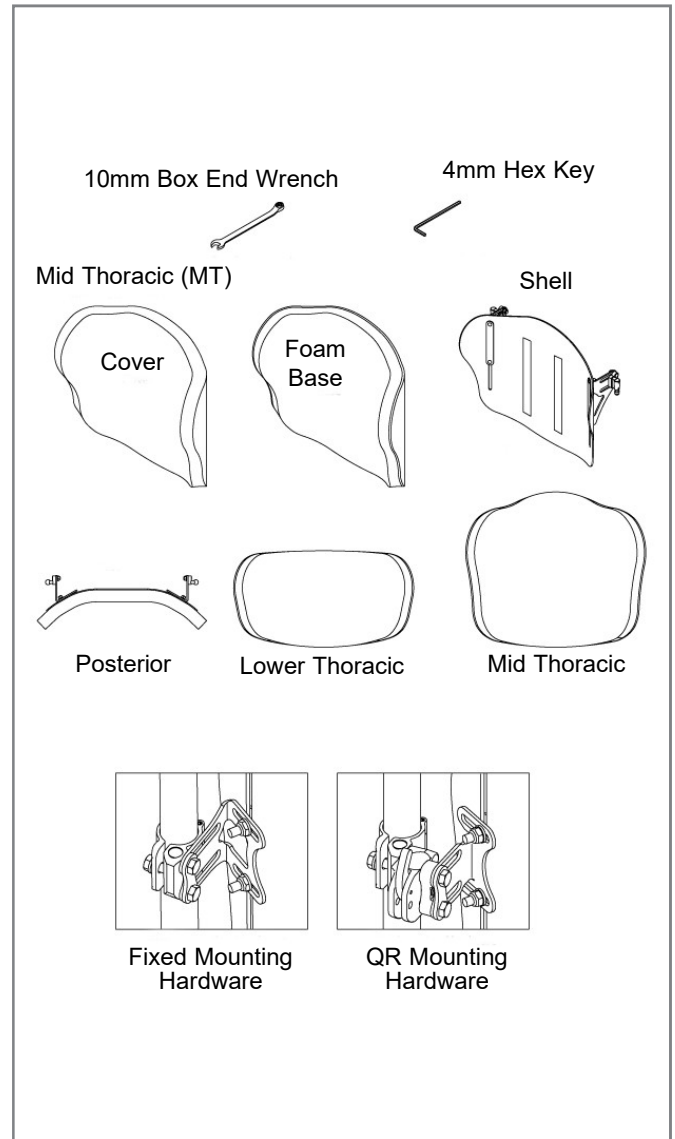MAINTENANCE

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to torque specifications as stated in the hardware installation section. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

EASY CLEANING

NOTE: When disassembling the back for cleaning, note the position of the Velcro® Spine Align Components for correct reassembly.

To clean back cover

Remove cover by detaching Velcro. Detach cover from back shell. Locate zipper on bottom of the cover and unzip. Remove foam and turn cover inside out. Zip cover closed to avoid snagging. Machine wash in warm water (60°C). Drip or tumble dry on low heat.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Reinsert the foam into the cover. Please note, the base has two layers of foam. When reinserting the foam, ensure that the thinner "light grey" foam is facing the shell. Once the foam is secured inside the cover, zip cover closed.

To clean back shell

Shell may be wiped clean with soap and water. No other maintenance should be required.

To clean positioning components

Positioning components can be wiped clean with soap and water. Do not immerse positioning components underwater. If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

CONDITIONS FOR SAFE STORAGE

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the "this side up" arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the "do not stack" mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

Laundry Care Symbols	
	Machine wash in warm water (60° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

DETERMINING COMPATIBILITY

Wheelchair Types

The J3 Carbon Back is designed to be compatible with most wheelchairs with the following exceptions: Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

WARNING!

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Wheelchair Dimensions

The J3 Carbon Back is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 1. Then using that measurement, check Table 1, (Fixed Mount) or Table 2, (Quick Release), to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter. The J3 Carbon Back hardware can be mounted on wheelchair back canes that measure in diameter 3/4" (19mm)(with insert), 7/8" (22mm)(with insert) or 1" (25mm) (no insert).

If you cannot determine that the back canes are compatible please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

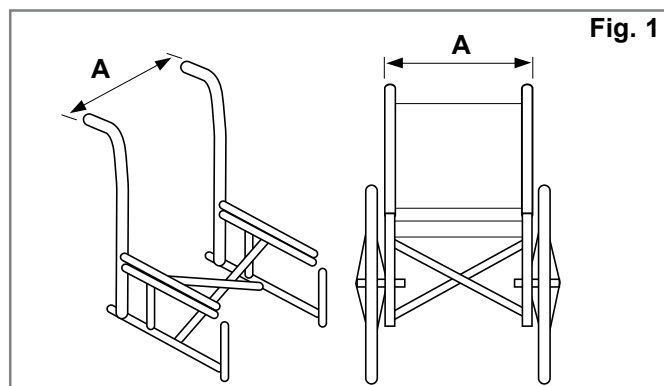
Fig. 1

Table 1

J3 CARBON BACK WIDTH Fixed Mount	Spacer	Width of back canes from outside to outside of the tubes, (Fig. 1-A)	
		Minimum Width	Maximum Width
32cm	NO	29cm	33cm
	YES	31cm	35cm
36cm	NO	34cm	38cm
	YES	36cm	40cm
41cm	NO	39cm	43cm
	YES	41cm	45cm
46cm	NO	44cm	48cm

Table 2

J3 CARBON BACK WIDTH Quick Release		Width of back canes from outside to outside of the tubes, (Fig. 1-A)	
		Minimum Width	Maximum Width
32cm		31cm	36cm
36cm		36cm	41cm
41cm		41cm	46cm
46cm		46cm	51cm

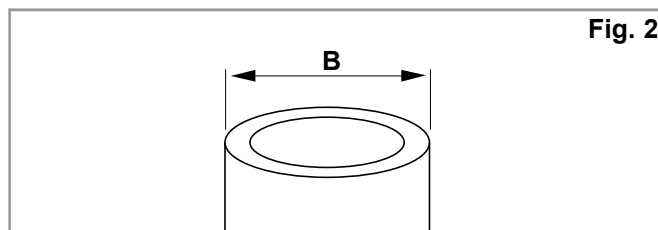
Fig. 2

Table 3

J3 Carbon Back Width	Cane Diameter (Fig.2)			
	Minimum Width		Maximum Width	
	Inch	cm	Inch	cm
12.6-18" (32-46 cm)	3/4	1.9	1.0	2.2

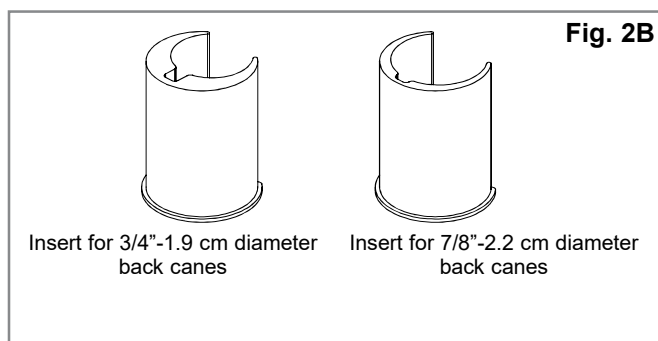
Fig. 2B

INSTALLING THE MOUNTING HARDWARE

Please read the following instructions before beginning installation.

To install the J3 Carbon Back, it is best to begin without a user in the wheelchair. Once the back has been installed and minor adjustments are needed the user can be seated back in the wheelchair.

Tools required (included with J3 Carbon Back)*

- 4 mm hex key, (only needed for the QR style)
- 10 mm Spanner.

NOTE: Wherever torque settings are specified, it is strongly recommended that a torque meter (not included with Jay3 Back) is used to verify correct torque specification is achieved.

Installation Instructions for the stationary mounting system, (Figs. 3, 4, 5).

1. Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.

NOTE: If your wheelchair width is 34cm, 40cm or 44cm wide, you will need to install the Spacers provided with your back: See Instructions below:

2. Using the included 10mm box end wrench, Loosen, but do not remove Nuts (C) and Screws (B).
3. Using the 10mm box end wrench loosen and remove screw (A), Open hinge to allow placement around the wheelchair Back Cane Tube.
4. Position the mounting hardware at the desired height on the Wheelchair's back canes and then close hinge and tighten hinge screw (A). The mounting hardware should be at approximately equal heights on each back post. Higher locations are generally better.




NOTE: for Back Cane tube diameters smaller than 25mm use the appropriate sized spacer.

5. Adjust the back to your desired position, depth, tilt and height and then tighten Screws (B) and Nuts (C).

Tighten all nuts and screws:

Recommended torque specifications:

- Screws (A) & (B): 11.3 – 12.4 Nm
- Nuts (C): 8.5 – 9.3 Nm.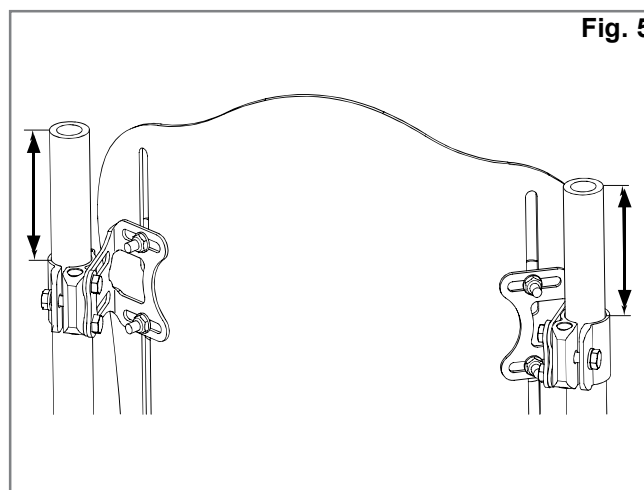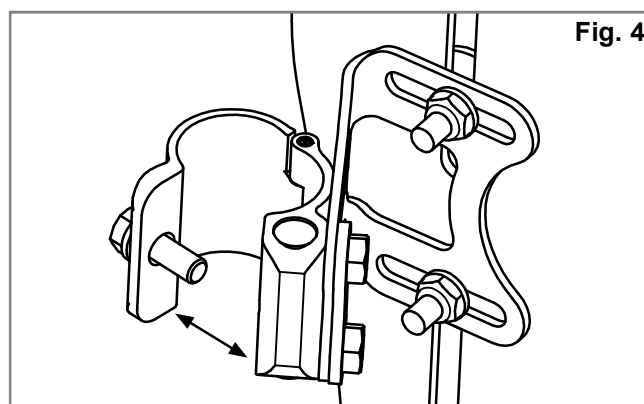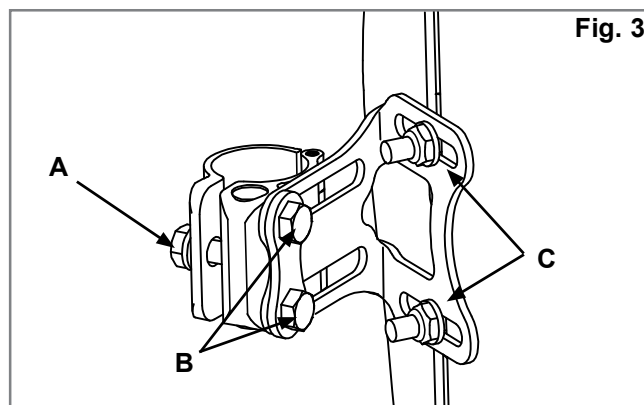

SPACERKITINSTALLATIONINSTRUCTIONS: (Fig. 6, 7).

NOTE: only needed for 34cm, 40cm & 44cm widths.

1. Remove the two (2) screws (B) that hold the Mounting Bracket to the Back Cane clamp assembly, (Fig.6).

NOTE: when you remove these screws, the barrel nuts will be loose. Be ready to catch them, so they do not fall to the floor.

2. Place the Spacer (D) between the Bracket and the clamp assembly.
3. Replace the Barrel Nuts if necessary and align holes to match up with the screws. Attach using the longer (M6 x 25) screws included in the kit.

INSTALLATION INSTRUCTIONS FOR THE QUICK RELEASE MOUNTING SYSTEM






(Fig. 8, 9, 10,).

1. Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
2. Visually determine the desired location of the receivers on the wheelchair's back canes. The mounting hardware should be at approximately equal heights on each back post (Fig. 8) and parallel to the seat frame, (Fig.9). Higher locations on the back canes are generally better.
3. Attach the hardware receivers. (Fig. 10)
Using the 10mm box end wrench loosen and remove screw (A), Open hinge to allow placement around the wheelchair Back Cane Tube.

NOTE: for Back Cane tube diameters smaller than 25mm use the appropriate sized spacer.

- Attach and align receiver to the wheelchair back canes.
- Hand tighten clamp screws (A).

NOTE: Receivers may require additional lateral adjustment to ensure proper alignment.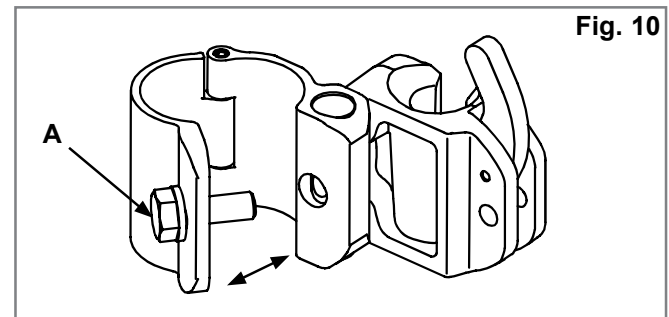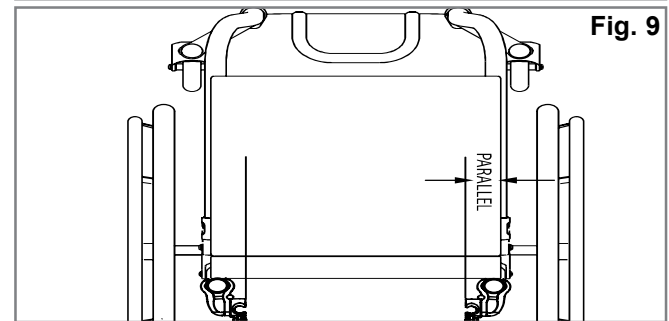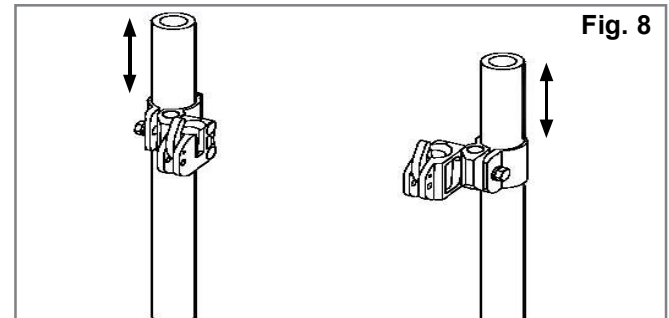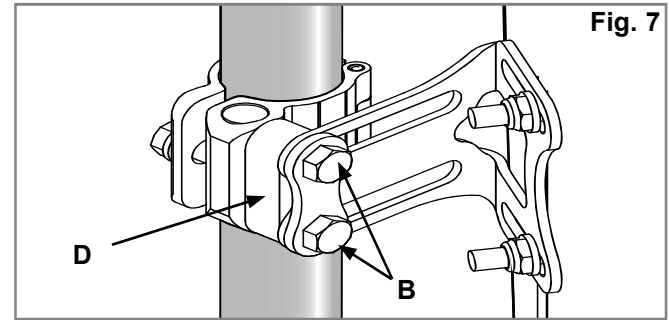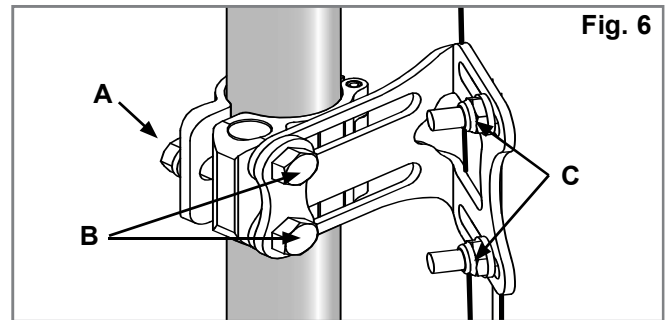

ATTACHING AND ALIGNING THE BACK SHELL

(Fig.11, 12, 13).

- Using the 10mm wrench, loosen but do not remove the Mounting Pin Screws (B) and Bracket Nuts (C).
- Attach the J3 Carbon Back to the Mounting hardware by inserting the Receiver Pins into the Receivers, (Fig.12).
- Adjust the back to your desired position, (Fig.13), depth, tilt and height and then hand tighten Screws (B) and Nuts (C).

NOTE: Back depth and angle may require additional adjustment to ensure proper fit to the user.

- Test back release by pressing the release levers forward and removing the back off the chair. (See Removal and Replacement section for further details.) Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
- If proper alignment has not been achieved, adjust the components until properly aligned.
- Once properly aligned, tighten all hardware.

NOTE: Tighten all nuts and screws only when the back is properly aligned:

Recommended torque specifications:

Screws (A) & (B): 11.3 – 12.4 Nm

Nuts (C): 8.5 – 9.3 Nm

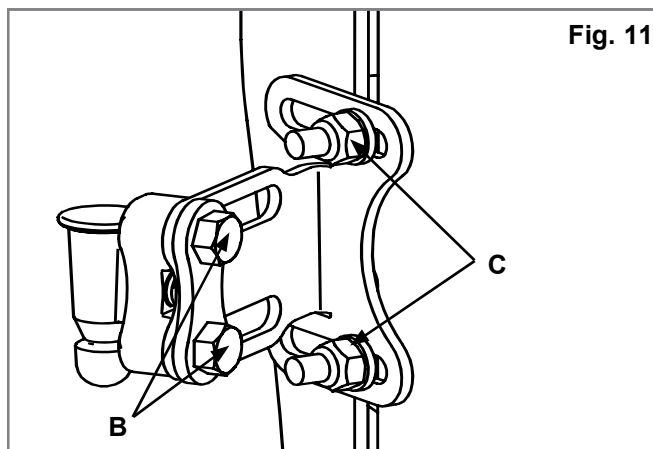
Fig. 11

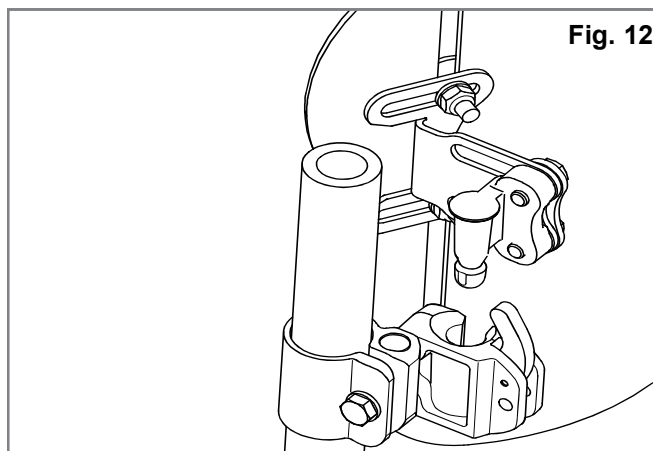
Fig. 12

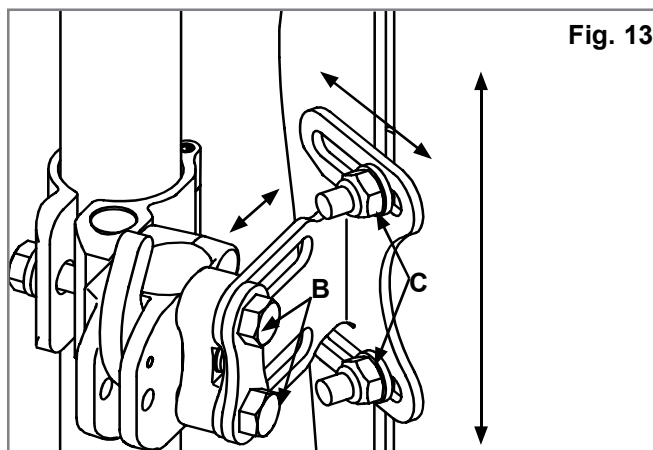
Fig. 13

ADJUSTING J3 HEIGHT, DEPTH AND ANGLE

The first phase of the J3 Carbon Back hardware installation is complete.

It is now the appropriate time to have the user seated back in the chair for personalizing the J3 Carbon Back. Do not have the user apply full pressure on the back as it is not fully tightened to the full torque requirements.

1. Height Adjustment (Fig. 14).

- With the user in the chair, identify the desired height of the back. Once identified, have the user remove pressure from the J3 Carbon Back by leaning or sliding partially forward if possible to allow the adjustment to take place.
- Loosen the bracket nuts (C) and adjust the height of the back shell to required position.
- Securely tighten the bracket nuts (C). The recommended torque specification is 75-85 in-lbs. (8.5-9.3 Nm).

NOTE: After height adjustment is complete, check that the width was not inadvertently changed.

2. Depth/Angle Adjustment (Figs. 11, 12, 13 & 14).

Please note there are two types of hardware that can be ordered— stationary hardware (Fig. 15) or quick release hardware (Fig. 16). Both sets of hardware are adjusted the same, but the stationary hardware allows for 1.6 inches (4 cm) of depth adjustment (Fig. 15) while the quick release hardware (removable hardware) allows for a total of 1.1 inches (2.9 cm) of depth adjustability (Fig. 16).

- To adjust J3 Carbon Back depth and angle loosen left and right screws (B) (Fig. 16) on bracket slots.
- Adjust depth and/or angle and securely tighten. The recommended torque specification is 100-110 in-lbs. (11.3-12.4 Nm). Ensure that none of the depth adjustment screws (Fig 16), (B), will slip or rotate.

NOTE: Prior to replacing the screws the application of a suitable medium strength threadlocker e.g. loctite 243 is strongly recommended to the receiver and screw threads, if the back is to be repositioned.




Recommended torque specifications:

Screws (A) & (B): 11.3 – 12.4 Nm

Nuts (C): 8.5 – 9.3 Nm

WARNING!

Ensure that all hardware has been properly tightened once all adjustments have been completed and the J3 Carbon Back is ready for use.

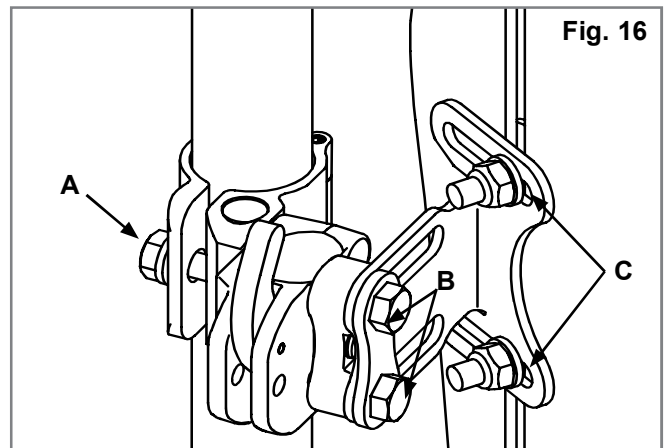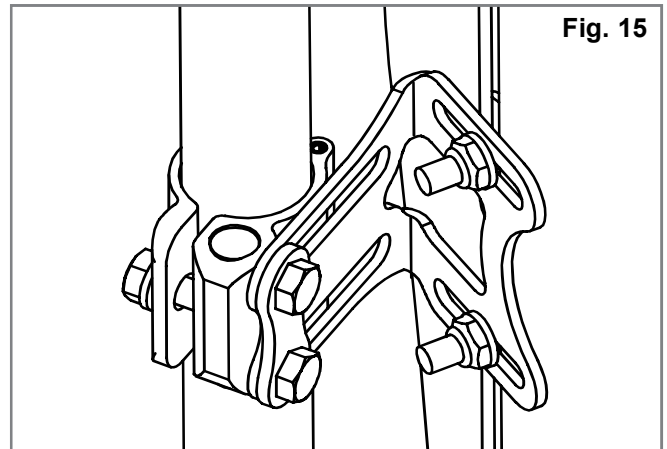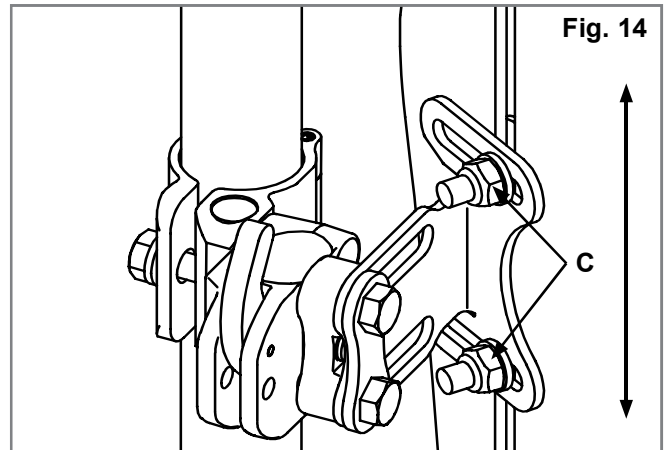REMOVAL AND REPLACEMENT

Removal – quick release hardware

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, push each release lever forward to unlock, (Fig. 17). Once both levers are in the forward position, simply lift the back straight up, (Fig. 18), using the handle located on the top of the back.

Proper reattachment after removal

To reattach the back to the hardware receivers, simply line up the pins to the receivers and push into place. If properly inserted the pins will “click” to a locked position, (Fig. 19-20).

WARNING!

If you are uncertain if you have heard the locking “click”, pull up on the back using the handle to ensure that it has been securely locked.

Removal – stationary hardware

If your J3 Carbon Backrest has a stationary hardware, follow the steps below to remove it from the wheelchair:

1. Loosen and remove the Hinge Screw(A) from the left and right clamp using 10mm wrench.
2. Open the Hinge and detach the J3 Carbon Back, (Fig.21).

Proper reattachment after removal To reattach the back, follow the installation instructions described in this manual (page 9).

Fig. 18

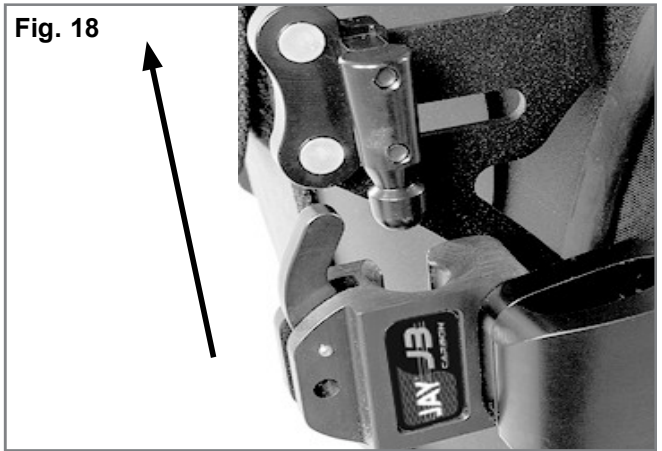
Fig. 19

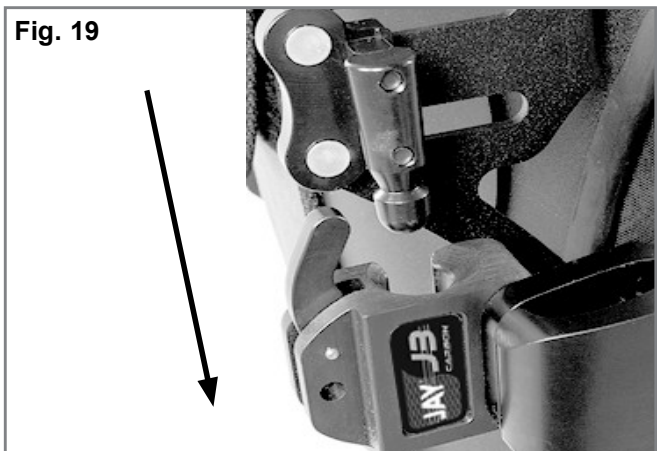
Fig. 20


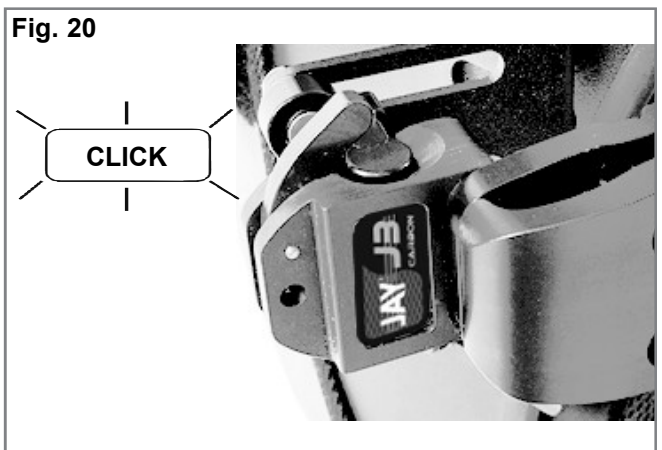
Fig. 17

Fig. 21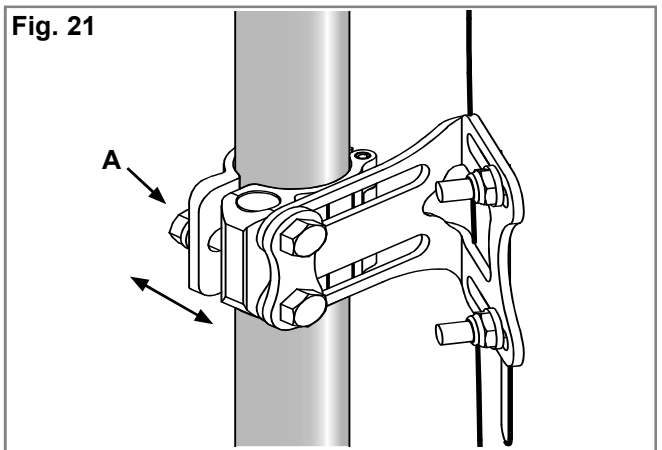

VANITY FLAP INSTALLATION

1. Lift upper flap of J3 Carbon Back Cover (Fig. 22)
2. Match hook to loop and attach Vanity Flap to the back of the J3 Carbon Back Cover (Fig. 23). Reposition upper flap of cover to sandwich the Vanity Flap between the flaps the upper and lower flaps (Fig. 24).
3. Remove Seat Back from wheelchair and sandwich "tail" of Vanity Flap between wheelchair support surface and Seat Back (Fig 25).
4. Adjust Vanity Flap as necessary by moving the tail in or out from under the Seat Back. If necessary the upper part of the vanity flap can be moved up or down on the Seat Back also.
5. Insert Hook & Loop Wrap ties through the appropriate grommets on the Vanity Flap, then wrap around the wheelchair frame and insert the leading end through the slot end of the Wrap (Fig. 26-27). Snug up and close outside gaps in Vanity Flap (Fig. 28).

Fig. 24

Fig. 25

Fig. 26

Fig. 27

Fig. 22

Fig. 23

Fig. 28

CONTRAINDICATIONS

WARNING!

Do not use or prescribe this backrest for anyone exceeding the stated Maximum User weight.

RECYCLING / RE-USE:

WARNING!

Risk of Injury.

Always have an authorized Sunrise Medical dealer inspect the product for damage or wear prior to transferring it to another user. If any type of damage is found, DO NOT USE THIS PRODUCT.

Failure to heed these warnings may lead to severe injury.

This Product is suitable for re-use*:

Have an authorized Sunrise Medical dealer inspect the shell, mounting hardware, foam and cover for damage or wear. If any damage is found, do not re-use.

Clean and disinfect the shell, mounting hardware foam and cover in accordance with local government regulations. Refer to your local Infection Control Office for advice.

After cleaning and disinfection, have an authorised Sunrise Medical dealer inspect the shell, hardware, foam and cover to insure that the cleaning/disinfection process did not cause any damage.

[- - - - -]
SAMPLE
[- - - - -]

JAY

Part Code:

Date:

cm
cm
kg

Max

Sunrise Medical (US) LLC
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA

Sunrise Medical GmbH
Kahlbachring 2-4
69254 Malsch / HD
Germany

Sunrise Medical Ltd.
Thorns Road, Brierley Hill
West Midlands DY5 2LD
United Kingdom

Assembled in Mexico

ISO 7010-M002
Instruction manual/
booklet must be read!
(Blue Icon)

Sunrise Medical Limited
Thorns Road, Brierley Hill
West Midlands, DY5 2LD
UNITED KINGDOM

106132 Rev. J

24 MONTH LIMITED WARRANTY

Each Jay J3 Carbon Back is carefully inspected and tested to provide peak performance. Every Jay J3 Carbon Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorised supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

Type:	Product Name/ SKU Number	XXXX-XX-XX	Date of Manufacture
Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
XXX mm	Seat width	MD	This symbol means Medical Device
XXX mm	Seat depth		Manufacturer's address
XXX kg	Max user weight	EC REP	European Authorised Representative
	CE Mark	UK RP	UK Responsible Person
	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

Indications

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury
- MS
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Muscle Dystrophy
- Amputation
- Tetraplegia
- Extremity defect deformity
- Joint contracture
- Cerebral Palsy

Contraindications

No contraindications are known or available.

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.SunriseMedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

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©2025 Sunrise Medical (US) LLC
Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S