Rolko
rainPRO

Regenschürze
Leg Protection
Protection de pluie
Voetenzak

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Dear Customer,

we thank you for your confidence in our company by purchasing one of our products. Our leg protection Rolko-rainPRO was developed according to the highest standards in order to make everyday life of people in need of care easier and more pleasant. All Rolko products were developed using the most advanced technologies and with the thoroughness and the team spirit of our employees. We are confident to support people in need of care and their helpers in the best possible way by offering a high value of benefit as well as good quality.

01. Purpose and Indication

Our leg protection covers the legs of wheelchair users against wind, rain and wet.

02. Contraindication

Our leg protection is not suitable for all applications that deviate from the intended use and/or may lead to danger.

03. Safety Instructions

- Please mind the instructions listed in this manual before using the product.
- Please check the leg protection regarding transport damages before it is used for the first time.
- An early deterioration of the product may be caused by influences of ultraviolet radiation, mechanical load as well as natural aging or careless handling, on which the producer does not have any direct influence.
- The water repellent and reflective side is worn outwards.
- Make sure that the leg protection or parts of it do not get into the spokes of the rear wheels.
- Fire hazard: Do not place the leg protection over warm or hot objects. Otherwise, the heat may melt the product material, causing damages. Keep open fire, such as burning cigarettes or ashes, away from the leg protection.

04. Use

First, use the parking brakes on the wheelchair so that it does not move.

Now simply put the leg protection over your legs.

Position your feet in the foot area.

Lastly attach the leg protection to the wheelchair frame with the press studs.

Detail: Press studs

05. Cleaning

If necessary, clean the leg protection with a damp cloth with the addition of a mild detergent. Hand wash is also possible. Do not use bleach, plasticizers or chemical cleaners. The leg protection is not suitable for dryers or washing machines.

06. Maintenance

Check the leg protection regularly and at least once a month for visual defects. Make sure the product is undamaged and the material is in perfect condition.

If there are signs of damage, we recommend the disposal of the leg protection.

07. Technical Data

Scope of delivery: 1 leg protection

Colours: outer fabric dark blue, inner fabric blue

Length: approx. 1.150 mm

Materials used: Outer fabric made of polyester with water-repellent PVC coating, inner fabric made of cotton, elastic bands with plastic press studs.

Disposal: The leg protection must be disposed of properly and professionally in accordance with the laws and regulations.

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

Alterations for reasons of technical development reserved. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances. Errors excepted.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Duitsland