Rolko
AIRpad

Antidekubitus-Sitzkissen
Anti-decubitus Seat Cushion
Anti Decubitus Zitkussen
Anti-decubitus Pude
Coussin de Siège Anti-décubitus
Cuscino sedile antidecubito

	Gebrauchsanweisung.....	Seite 02-05
	Instruction manual.....	Page 06-09
	Gebruiksaanwijzing.....	Pagina 10-13
	Betjeningsvejledning.....	Page 14-17
	Mode d'emploi.....	Page 18-21
	Istruzioni per l'uso	Pagina 22-25

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CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen

Thank you very much for buying the Rolko-AIRpad and your trust in our products. Please pay attention to all safety instructions before using the cushion the first time. Check package/article for completeness.

01. Product description / Suitability

Rolko-AIRpad provides the feeling of firm and stable sitting in a wheelchair and other seat devices. It prevents soreness and helps the healing process of decubitus. Rolko-AIRpad can replace other already existing wheelchair cushions. The combination of visco-elastic foam and adaptive air-chamber provides pressure relieve and comfort for sensible areas at prolonged sitting. The cushion can be used for prophylaxis and for decubitus therapy up to stage II, in accordance with EPUAP. The full-contour air-chamber of the cushion with adjustable air volume will provide a custom fit for every user. User weight is evenly distributed over the entire seat area, reducing soreness in sensitive areas. At the same time, the memory-foam inside the air bag has a stabilizing effect, supporting a proper posture for the user. The two-way-valve allows at any time a quick simple adjustment of the cushion to the user. The cushion is self-inflating; no air-pump required! The size of the cushion must be chosen according to the dimensions of the seat (see size table under Technical Data). The maximum user weight is 150kg.

02. Contraindication

Above 150 kg or under 30 kg, the seat pad may not be used. A person who is unable to sit may not use the cushion. Rolko-AIRpad should also not be used in combination with other cushions or covers. Users with incontinence should only use the cushion together with optional incontinence cover. As soon as you realize symptoms of skin-rash or other irritations, which do not disappear within a short time after relief of the damaged skin areas, stop using it. Contact your medical doctor, therapist or your health care supply store.

03. Safety Instructions

- Dealer: This instruction manual has to be handed out to the customer. Please make sure that the user of the Rolko-AIRpad cushion fully understands this product information.
- User: Read this instruction manual carefully for safe use and full benefit. Keep this instruction manual at hand for re-use of the seat pad.
- Check regularly for skin rash and other irritations especially in the area of the ischial tuberosity and the coccyx. If the skin-rash or irritation persists, please contact your medical doctor immediately.
- Do not use the seat cushion in combination with other cushions, original seat pad covers and only in correct size, the seat cushion as floating aid and do not use a pump for inflation; - the cushion is self inflating!
- Respect user weight limitation!
- Keep the seat cushion away from heat sources, hot ashes, cigarettes and flames! Do not expose the cushion to direct sunlight for an extended time.
- Keep the cushion away from sharp and pointed objects, to avoid damage of the air chamber!
- Do not carry the cushion at the valve stem!
- Do not put any other objects between seat face and AIRpad nor between AIRpad and user. This will impair proper function of the AIRpad!
- The Rolko-AIRpad cushion should only be used with the proper air volume:
 - Under-inflation may cause injury to the soft tissue.
 - Over-inflation prevents the user from sinking into the cushion, which strongly reduces the positive effect of the cushion and may result in injuries of skin and soft tissue.
- Follow the cleaning instructions for seat pad and cover. Keep the valve closed at any time during the cleaning process!
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH, as well as to the competent authority of the EU member state in which the user is established.

04. Application

Remove all cushions and potential other items from the wheelchair seat or other sitting device. Place the Rolko-AIRpad with the grey, anti-slip bottom side onto the seat. The breathable netlike top side of the cover points upward. Turn the Rolko-AIRpad until the valve points to the front and make sure it shows through the opening in the seat cover.

Both of the grey surfaces - seat pad and cover - have to point downwards! In this case, by looking at the valve stem, the white dot at the seat pad is on the left.

Adapt the seat pad:

1
Valve open / inflating

1. Open valve

Put the seat cushion without load on the seat. Open the valve by turning it clockwise. The white marker on the valve cap now points to the top and the cap noticeably snaps in position. Make sure that the cover is not caught by the valve cap! The cushion now inflates itself. It is fully inflated, once the hissing noise at the valve stops.

2
valve closed

2. Close valve and seat

Now turn the valve counter-clockwise to close it. Once fully closed, the white marker at the valve cap points towards the white dot of the cushion and the cap snaps into position. Sit down, or have the user sit down on the cushion, while the valve is still closed. Make sure, arm- and foot-rests of the wheelchair are properly positioned.

3
Deflating

3. Comfortable adjustment

Open the valve again by turning the cap clockwise, until you overcome a noticeable resistance. If you now slightly turn the valve cap further, some of the air escapes from the cushion. Let only so much air escape, until the seating position feels stable and comfortable on the cushion. Any bone part of the pelvis should not touch the hard seat surface. Recommended optimal distance between lowest point (usually ischial tuberosity) and bottom side of cushion is approximately 1,5- 2 cm.

As a safety precaution check proper air volume of the cushion regularly:

As a simple measure, put your hand between the cushion and the lowest point of your buttocks. If you can move your fingers just a little, you have the proper inflation.

In case you have to re-adjust firmness of the cushion, always start again with point 1 of the instructions.

 **Check regularly proper filling of the air cushion. Make sure the cushion offers enough padding to avoid any pressure point sores, but is also not too firm when over-inflated!**

Removing and fitting the cover

Removing the cover:

1. Load the cushion, release air from the cushion by turning it clockwise and close the valve again.
2. Open the zipper and pull the cover carefully over the cushion. Avoid twisting or bending the valve!

a
Bottom of cover and cushion lie on top of each other

b
Slot for valve in cover

c
Pocket for zipper puller

Using the cover:

1. Load the cushion, let air out of the pillow by turning it clockwise, close the valve again and open the zipper of the cover.
2. Open the zipper and insert the cushion with the valve ahead into the cover. Make sure, the grey side of the cushion is facing the grey anti-slip side of the cover (a).
3. Pull the cover fully over the cushion and put the valve through the slot of the cover (b). Close the zipper and put the puller into the protection pocket at the end (c).
4. Now open the valve to allow the seat cushion to self-inflate. Check if the cover is fitted properly and without wrinkles.
5. The correct air volume must be set before use according to „04. application“.

05. Cleaning and Desinfection

For cleaning and desinfection remove the cover from the cushion as explained above.

Cushion:      Hand wash only | do not tumble dry | do not iron | do not dry clean | do not bleach

Always close first the valve before you start cleaning the cushion, to avoid that any water is getting inside the cushion. The valve is closed when white dot and white mark on the valve face each other.

Little dirt of the seat cushion can be cleaned with a damp cloth, if necessary by use of a mild detergent (e.g. mild soap). For heavy dirt the cushion can be hand-washed. For desinfection use a microfibre cloth, soaked with a mild alcohol based desinfectant solution. Alternatively, an alcohol based spray desinfectant can be used. Please let the cushion dry at ambient. Do not iron the cushion. Don't use bleach, softener or any chemical dry-cleaning agents!

Take care, that the valve is always closed before cleaning!

Standard cover:     Max. 60°C | dryable (gentle) | do not iron | do not dry clean | do not bleach

The standard cover can be machine washed at 60°C with a gentle wash cycle and by using a mild detergent. For desinfection use a microfibre cloth, soaked with a mild alcohol based desinfectant solution. Alternatively, an alcohol based spray desinfectant can be used.

To protect other pieces of cloth during the wash cycle we recommend to close the zipper of the cushion and use a string bag. Do not iron the standard cover. Do not use bleach, softener or any chemical dry cleaning agents!

Thereafter, the cover can be dried at ambient or tumble dried at low temperature.

After all components are completely dry, fit the cover again to the cushion as explained under removal / fitting of the cover.

Consider the wash label at the cover!

06. Maintenance

To maintain full functionality of the seat pad, let the cushion relax over night once a week with opened valve. Check the seat cushion regularly for tightness and proper functionality of the valve. If the cushion cannot hold the air, the air chamber may be punctured; you may also notice a hissing noise. Check the entire contact area of the seat cushion top and bottom fabric and of the valve regularly. Look for damages to the surface, for abrasion and small holes. Check periodically all seams and for a proper fit of the cover to avoid the development of folds.

07. Re-use

The seat cushion can be re-used. Before using the seat cushion for another patient, please perform the cleaning and desinfection steps according the 05. cleaning and desinfection procedure. Please make sure, that the AIRpad is passed on to the next user with all the components. This instruction manual is part of the the package and must be forwarded together with the cushion to the next user.

Stop using a seat cushion with visible damage!

08. Disposal

The Rolko-Airpad is to be disposed of safely with household waste.

09. Technical data

Used materials:

Single-chamber air-cushion : 75% Polyester, 25% Thermoplastic Polyurethane

Foam: 100% Polyurethane memory foam

Valve: 40% Polyurethane, 30% Nylon, 30% Polyoxymethylene

Standard cover: 100% Polyester

Anti-slip-bottom: 70% Polyester, 30% Polyurethane

Optional accessories:

Incontinence cover: 80% Polyester, 20% Thermoplastic Polyurethane

Instruction Manual

Permissible environmental conditions:

Ambient temperature: -25 up to 110°C

Min. user weight: 30 kg

Max. user weight: 150 kg

Max. pressure in the cushion corresponds to ambient air pressure

Delivery content:

1 seat cushion

1 removable standard cover

1 instruction manual

Optional accessory: incontinence cover

standard cover

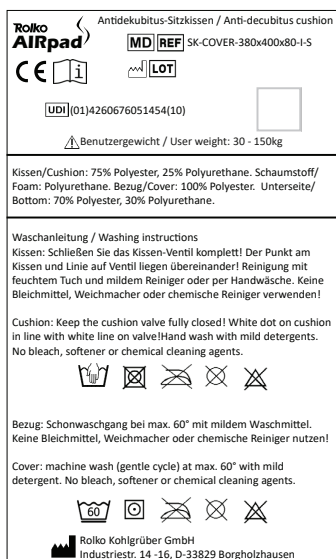
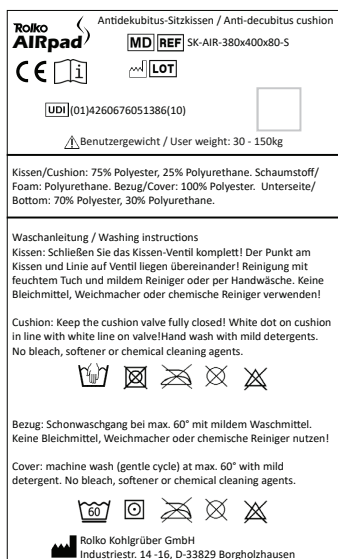
seat cushion

Dimensions:

Item no.	Item no. (Incontinence Cover)	Width (valve side)	Depth	Height	Net weight
SK-AIR-380x400x80-S	SK-COVER-380x400x80-I-S	38 cm	40 cm	8 cm	680 g
SK-AIR-400x400x80-S	SK-COVER-400x400x80-I-S	40 cm	40 cm	8 cm	720 g
SK-AIR-430x400x80-S	SK-COVER-430x400x80-I-S	43 cm	40 cm	8 cm	760 g
SK-AIR-430x430x80-S	SK-COVER-430x430x80-I-S	43 cm	43 cm	8 cm	830 g
SK-AIR-450x400x80-S	SK-COVER-450x400x80-I-S	45 cm	40 cm	8 cm	800 g
SK-AIR-450x450x80-S	SK-COVER-450x450x80-I-S	45 cm	45 cm	8 cm	860 g
SK-AIR-480x430x80-S	SK-COVER-480x430x80-I-S	48 cm	43 cm	8 cm	880 g

Identification:

-  Manufacturer
-  Follow the instructions
-  Production date
-  Item no.
-  Batch
-  Medical device
-  Unique device identification
-  Hand wash only
-  Machine wash up to max. 60°C
-  Do not tumble dry
-  Tumble dry suitable (gentle)
-  Do not iron
-  Do not dry clean
-  Do not bleach

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN 12182. The EU declaration of conformity is available for download at instructions.rolko.com.

Operating Instruction:  | Anti-decubitus seat cushion | 2024-04-11 | Rev. 13
Basis-UDI-DI: 426067605AIRPAD000000004P

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen