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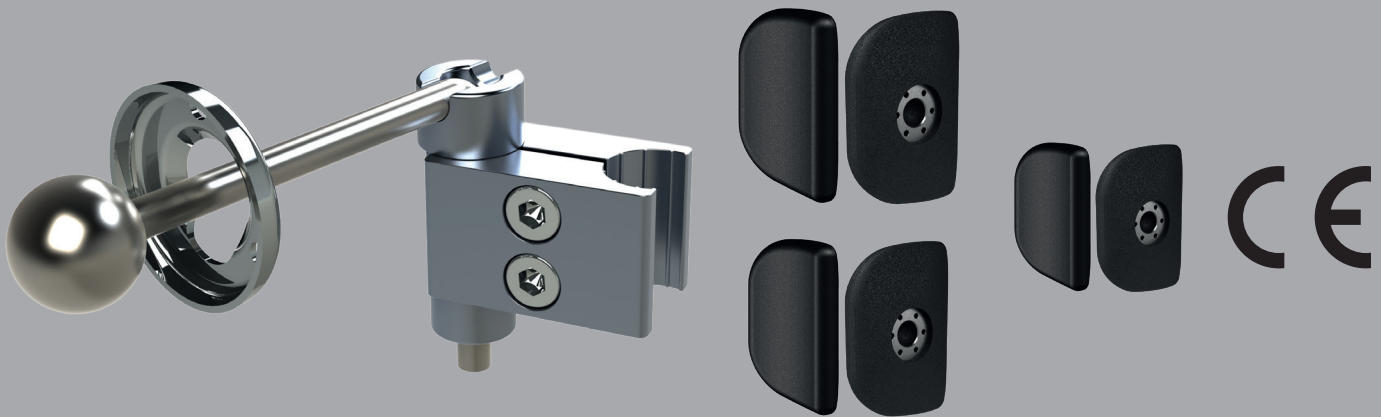
Dear Customer,

the Rolko team thanks you for the trust you have placed in us by purchasing our product. With the purchase of the side pad holder system, you have chosen a highly functional product that makes everyday life easier for those in need of care. Our side pad holder system was developed taking into account the current standards, using the latest technologies and with the thoroughness and team spirit of our employees. We are sure that we provide the best possible holder to those in need of care with our high utility value and solid quality.

01. Purpose and Indication

These instructions for use are intended for specialists. The side pad holder systems and the foldable side pads from Rolko offer assistance in positioning the upper body, e.g. of people with disabilities or patients in wheelchairs. The side pad holder systems provide the right hold and still offering plenty of space to move.

02. Contraindication

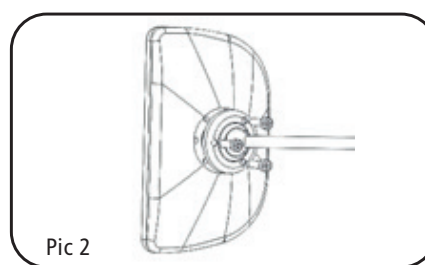
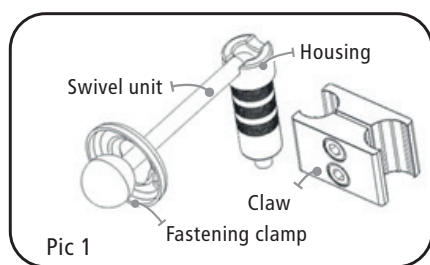
The necessity of use and the suitability of the wheelchair for the attachment of the side holder systems are to be evaluated by specialists. Side pad holder systems with swivel rod can only be mounted on frames with a round tube diameter of 22 or 25 mm. The holder systems cannot be mounted on wheelchairs with other tube diameters.

03. Safety Instructions

- These instructions must be read and followed before installing the product.
- Check the side pads and the holders for damages before using them for the first time.
- Damaged and or defective side pads or holders must never be used and need to be replaced immediately.
- The instructions for use are part of the side pads or holders and must be passed on.
- If serious incidents occur in connection with this product, these must be reported to Rolko Kohlgrüber GmbH and to the competent authority of the EU Member State in where the product is being used.

04. Assembly and Description

The side pad holder systems with swivel unit (Pic 1) consist of a housing in which the swivel arm is springing mounted and the fastening clamp for the side pad itself. The claw made of aluminium connects the swivel unit to the frame of the wheelchair. The scope of delivery also includes the 3 Allen screws DIN 6912-M5x16 for attaching the side pad to the ball head (Pic 2).

The side pad holder system with swivel rod is attached to the frame of the wheelchair at the correct height. The possibilities of use and the correct position are to be determined by trained specialists.

The swivel arm can be bent into a desired or required shape with a suitable bending tool. The side pad attached to the side pad holder system with swivel arm supports the trunk of the user or patient in the wheelchair. The side pad can be swivelled to the side to get in or out of the wheelchair. To do this, pull the swivel arm upwards out of the housing until the groove releases the swivel arm. Then the swivel arm with the side pad can be turned away. In the groove offset by 180 °, these can then be clicked into place if necessary.

The side pads PU-SP-165x100-S, PU-SP-205x125-S or PU-SP-235x150-S can be mounted on the holding systems with swivel arm.

Item no.	Swivel arm with side pad holder	Claw for wheelchair frame	Photo
U-BE-SP95-HA22	U-BE-SP95 Length: 95 mm Ball Ø: 25 mm	HA-BRF-ALU40-22 For tube Ø: 19/22 mm	
U-BE-SP95-HA25	U-BE-SP95 Length: 95 mm Ball Ø: 25 mm	HA-BRF-ALU40-25 For tube Ø: 19/25 mm	
U-BE-SP165-HA22	U-BE-SP165 Length: 165 mm Ball Ø: 25 mm	HA-BRF-ALU40-22 For tube Ø: 19/22 mm	
U-BE-SP165-HA25	U-BE-SP165 Length: 165 mm Ball Ø: 25 mm	HA-BRF-ALU40-25 For tube Ø: 19/25 mm	

Side pad models

05. Cleaning and Disinfecting

Wipe off the side pad holder systems with a damp cloth, if necessary with the addition of a mild detergent. Use lukewarm water and, if necessary, non-abrasive, pH-value neutral, commercially available household cleaners. If necessary, the side pad holder systems can be cleaned with common household disinfectants by spray or wipe disinfection. The use of cleaning agents and disinfectants with chlorine or organic or inorganic substances with active chlorine is not permitted. The following cleaning agents and disinfectants or processes are not permitted for the side pads (made of polyurethane):

- hot water, steam,
- concentrated acids and alkalis,
- alcohol,
- chlorinated hydrocarbons,
- ammonia.

06. Maintenance

Verify the function of the side pads as well as the holders before each use and make sure that they are undamaged and in perfect condition. Before each use, check:

- the tight fit of all fixations,
- the correct adjustment of the side pad holders,
- the products visually for any damages.

07. Re-use


This product is suitable for re-use. Clean the side pads and holders according to the specifications under "05. Cleaning and disinfection" and pay attention to the indications regarding the maintenance above.

08. Disposal

All metal parts of the side pads and side pad holder systems are accepted by the recyclable materials trade. The side pads made of polyurethane can be safely disposed of with household waste. If possible, dismantle the side support holding system into its individual parts to enable the materials to be sorted.

09. Technical Data

Item no.:	U-BE-SP95-HA22, U-BE-SP95-HA25 U-BE-SP165-HA22, U-BE-SP165-HA25
Scope of delivery:	side pad holder system, instruction manual side pads (optional): PU-SP-235x150-S, PU-SP-205x125-S, PU-SP-165x100-S
Swivel range:	180°
Temperature range:	-20°C up to +50°C relative humidity - max. 80%
Materials used:	aluminium, stainless steel, steel, polyurethane
Labelling:	

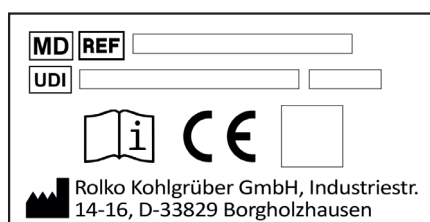Manufacturer

Follow the instructions

Item no.

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. The EU declaration of conformity is available for download at www.rolko.com.

Instruction manual: Side pad holder systems - U-BE-SP95-HA22, U-BE-SP95-HA25, U-BE-SP165-HA22, U-BE-SP165-HA25

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. Alterations for reasons of technical development reserved. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances. Errors excepted.

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01. Purpose and Indication

These instructions for use are intended for trained specialists. The side pad holder systems and the foldable side pads from Rolko offer assistance in positioning the upper body, e.g. of people with disabilities or patients in wheelchairs. The side pad holder systems provide the right hold and still offering plenty of space to move.

02. Contraindication

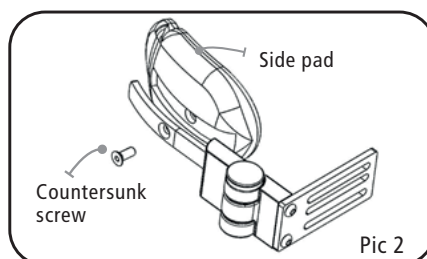
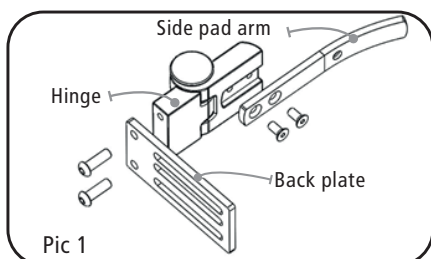
The necessity of use and the suitability of the wheelchair for the attachment of the side holder systems are to be evaluated by trained specialists. The side pad holder systems with hinge can only be mounted on wheelchairs with a fixed back and on suitable seat shells. The holders are unsuitable for other wheelchairs.

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

04. Assembly and Description


The side pad holder systems with hinge consist of the hinge, the back plate and the pad arm (see pic 1), which are screwed together. The side pad PU-SP-130x85-S-M6 or PU-SP-165x100-S-M6 attached (see pic 2). The side pad PU-SP-130x85-S-M6 or PU-SP-165x100-S-M6 is attached to the pad arm with a countersunk screw M6x16 (see pic 2). The countersunk screw M6x16 is not included in the scope of delivery.

The side pad holder system is screwed at the correct height to the fixed back of the wheelchair or to the back of the seat shell. The fixation material required for this is not included in the scope of delivery. The possibilities of use and the correct position are to be determined by trained specialists. The side pad attached to the holder system supports the torso of the user or patient in the wheelchair. The side pad can be folded down to get in or out of the wheelchair. To do this, press the push button on the hinge (see pic 3) and swivel the side pad to the side (see pic 4).

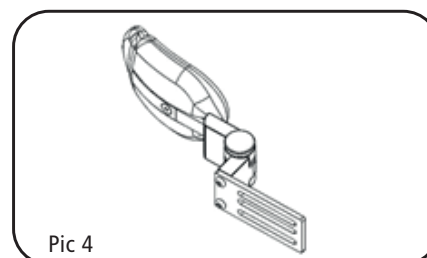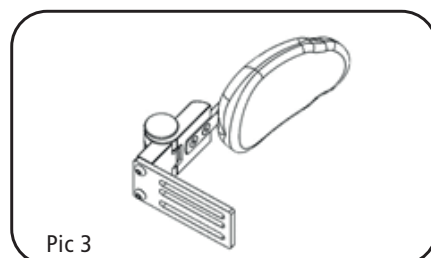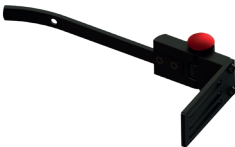The HA-SP-ALU-5 and HA-SP-ALU-6 hinges are more compact and shorter. The side pad holder systems for which these hinges are to be used are therefore more suitable for wheelchairs for children. Only the side pads PU-SP-130x85-S-M6 or PU-SP-165x100-S-M6 can be mounted on the side support systems with hinge.

Item no.	Hinge	Back plate	Side pad arm	Photo
HA-SP-ALU-100-SL130	HA-SP-ALU-2	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-130	
HA-SP-ALU-100-SR130	HA-SP-ALU-1	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-130	
HA-SP-ALU-100-SL200	HA-SP-ALU-2	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-200	
HA-SP-ALU-100-SR200	HA-SP-ALU-1	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-200	
HA-SP-ALU-J-100-SL130	HA-SP-ALU-6	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-130	
HA-SP-ALU-J-100-SR130	HA-SP-ALU-5	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-130	
HA-SP-ALU-J-100-SL200	HA-SP-ALU-6	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-200	
HA-SP-ALU-J-100-SR200	HA-SP-ALU-5	HA-SP-ALU-RB-100x45	HA-SP-ALU-SL-200	

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06. Maintenance

Verify the function of the side pads as well as the holders before each use and make sure that they are undamaged and in perfect condition.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen