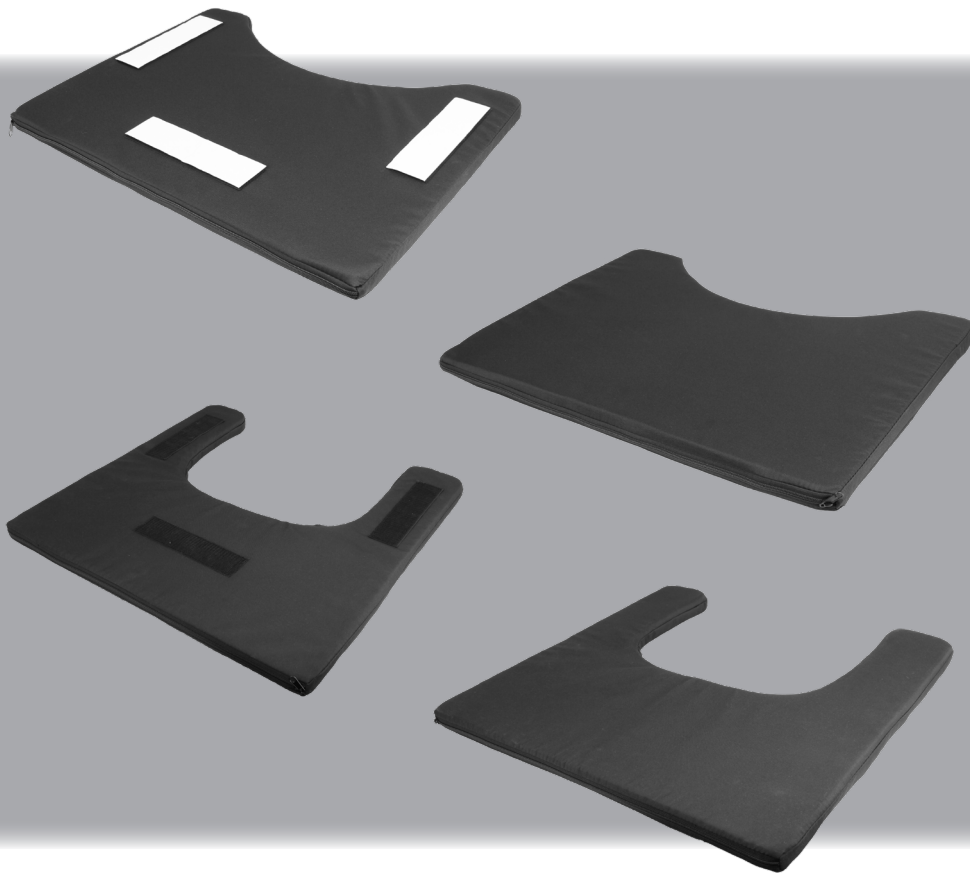
Therapietischpolster

Therapy Table Pads

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Polstring til terapibord

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01. Purpose and Indication

The cushions for therapy tables are used together with a therapy table. They protect the patient from injuries during uncoordinated movements. The respective cushion must be chosen according to the dimensions of the therapy table top.

02. Contraindication

Each cushion for therapy tables may only be used for the appropriate therapy table. Do not use liquids, strong heat sources, cigarettes, or open flames when using the cushion. No additional covers, cushions, or pillows may be used.

03. Safety Instructions

- This user manual must be read and observed before use.
- Each cushion for therapy tables may only be used for the appropriate therapy table. See Section 04. Assembly.
- Do not use liquids, strong heat sources, cigarettes, or open flames when using the cushion.
- No additional covers, cushions, or pillows may be used.
- This user manual is part of the cushion for therapy tables and must be passed on when it is handed over.
- In the event of serious incidents related to this product, they must be reported to Rolko Kohlgrüber GmbH and the competent authority of the EU member state in which the user is established.

04. Assembly

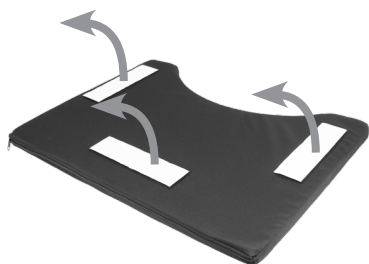
Figure 1: Remove the protective strip from the Velcro tape.

Figure 2: Stick the cushion appropriately onto the therapy table.

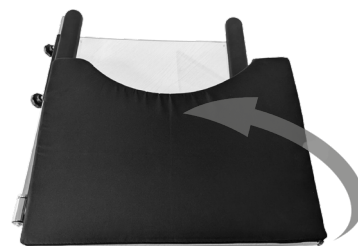
Figure 3: Separate the Velcro and remove the cushion. One side of the Velcro tape remains on the therapy table.

- Ensure that your cushion for therapy tables is suitable for the intended therapy table.
- Remove the protective strips from the Velcro tape (Figure 1).
- Place the cushion appropriately on the therapy table top and press the adhesive strips of the Velcro tape firmly (Figure 2).
- Now the cushion can be removed by carefully separating the Velcro tape (Figure 3).
- You can use the therapy table with or without the cushion as needed.

05. Cleaning and Disinfection

The cover is water-repellent but not completely waterproof. In most cases, it is sufficient to wipe the cover with a damp cloth, if necessary, using a mild cleaner. For heavy soiling, remove the cover and clean it as follows:

Max. 60°C | gentle drying | do not iron | do not dry clean | do not bleach

If liquids or dirt have penetrated the cushion (PU foam), rinse it with clear water, if necessary, using a mild cleaner. Allow the cushion to air dry. The cover can be disinfected with special disinfectants for textiles. Please observe the instructions for use of the disinfectant.

Instruction Manual

06. Maintenance

Before each use: Check the fit of the cushion on the therapy table and the adhesion of the self-adhesive Velcro tape.

Once a month: Check the seams of the cushion and the cover for dirt and defective seams.

If the adhesive effect of the self-adhesive Velcro tape on the therapy table is no longer sufficient, it must be replaced. For dirt, clean the cover according to Section 05. Cleaning and Disinfection. If there are defective seams and tears in the cover, the entire cushion must be replaced.

07. Re-use

The cushion for therapy tables is suitable for reuse. Before using the cushion for another patient, please clean it as described in Section 05. Cleaning and Disinfection. This user manual is part of the delivery and must be passed on when the cushion is handed over or reused.

08. Disposal

Remove the cover. Both parts can then be safely disposed of with household waste.

09. Technical Data

Item no.: POL-TP-L1, POL-TP-L2, POL-TP600-KLP
Material: Polyester, PU
Suitable for: See table below

Item no. Pad	Item no. Therapy table	Width	Depth	Height	Weight
POL-TP-L1	TP-L1	60 cm	57 cm	3,3 cm	Approx. 380 g
POL-TP-L2	TP-L2	67,5 cm	57 cm	3,3 cm	Approx. 405 g
POL-TP600-KLP	TP600-KLP-3	60 cm	45 cm	3,3 cm	Approx. 370 g

Label

Manufacturer

Follow the instructions

Production date

Item no.

Batch

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. The EU declaration of conformity is available for download at rolko.com.

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen