quick2go®

CE

**Türschwellerampe
Door Threshold Ramp
Drempelhulp
Dørtrinsrampe
Passage de Seuil
Rampa per soglia**

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Instructionmanual.....Page06 - 09

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Brugsanvisning Side 14 - 17

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CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen

Thank you very much for purchasing our door threshold ramp and for your trust in ROLKO products. Before installing and using this product, please carefully read the entire user manual. Check that all parts are included and follow all of the safety instructions.

01. Application / Purpose

The threshold ramp quick2go allows the overcoming of door thresholds, steps and other obstacles. Suitable for wheelchairs, e-chairs, electric scooters, etc. with a total weight up to 250 kg. Based on the height and nature of the obstacle you receive the appropriate number of wedges, blocks and clips for the ramp. These instructions for use are aimed at users of the ramps and support staff.

02. Contraindication

The maximum load capacity of 250 kg must be observed for the threshold ramp quick2go. The track width of the device must not exceed 700 mm. The maximum permissible height is 77 mm.



03. Safety Instructions

- The ramps is to be used only at a moderate speed, please avoid sudden acceleration or deceleration.
- Be very careful while using the ramp during strong weather conditions like snow, frost or wetness.
- The non-slip material of underside of the ramp has to rest evenly.
- In the case of PVC and vinyl floor coverings, direct contact with the door sill ramp may cause discoloration of the floor covering. Therefore, avoid direct contact!
- When using the glue, observe all safety instructions for the cyanoacrylate glue! Avoid skin and eye contact! When processing indoors, ensure a good ventilation!
- The instructions for use are part of the threshold ramp and must be handed over with it.
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH, as well as to the competent authority of the EU member state in which the user is established.

04. Assembly/Use

The threshold ramp quick2go can be modularly build in several layers, according to the obstacle height. Based on the following pictures you can see how the ramp parts are mounted:

1. For the first layer lay out the required number of rubber parts. The arrows on the upper side of the rubber blocks show the obstacle direction (see fig. A). Connect all elements from below with the clip connectors VB1 (see fig. B).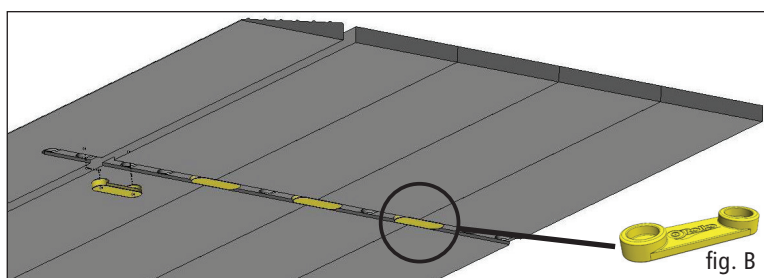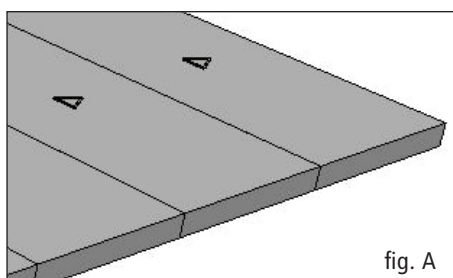

2. Insert the clip connectors VB2 from above, observe the correct orientation (fig. C + D)! Connect all rubber parts lying together and additionally insert in the last two holes of the rubber part a clip connector VB2.

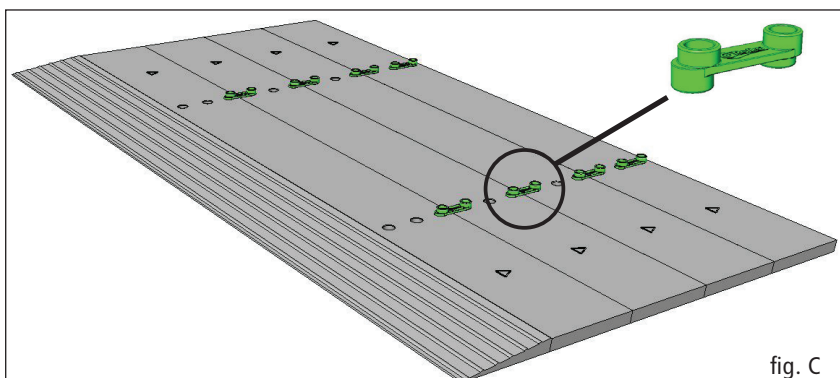
fig. C

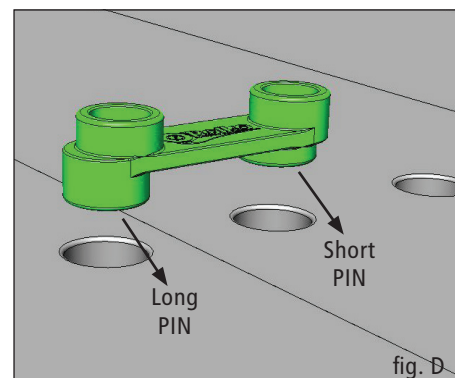
fig. D

3. For the second ramp layer insert another layer of rubber parts on the already attached clip connectors. Continue mounting the ramp alternating with rubber parts and clip connectors as described in step 2 (see fig. E).

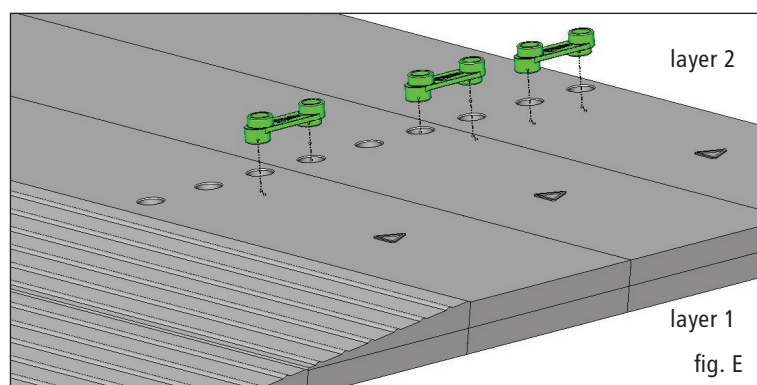
fig. E

4. On the topmost rubber block connect 2 clip connectors VB2 in reversed position (see fig. F + G).

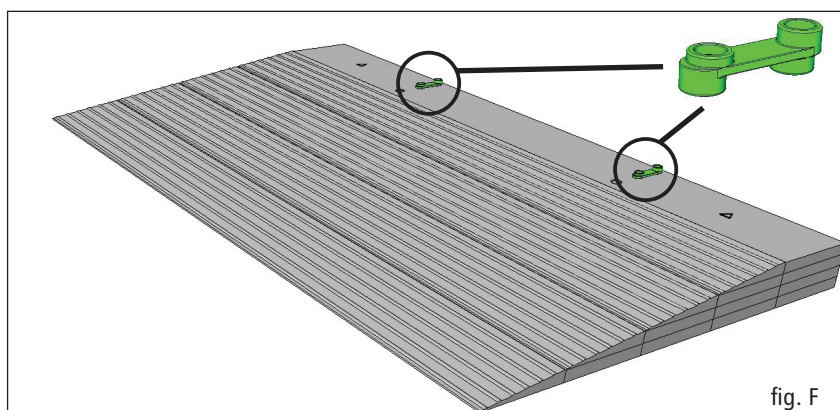
fig. F

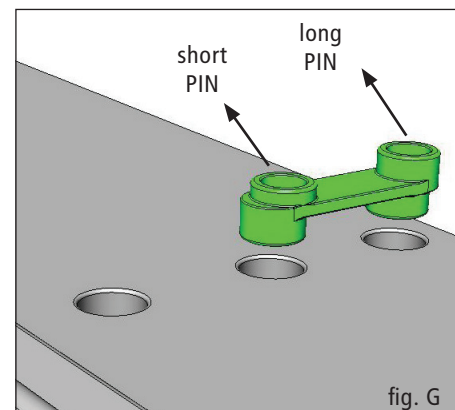
fig. G

5. After that place the top and last rubber wedge on the ramp and press it firmly. The door threshold ramp quick2go is now ready for use (see fig. H).



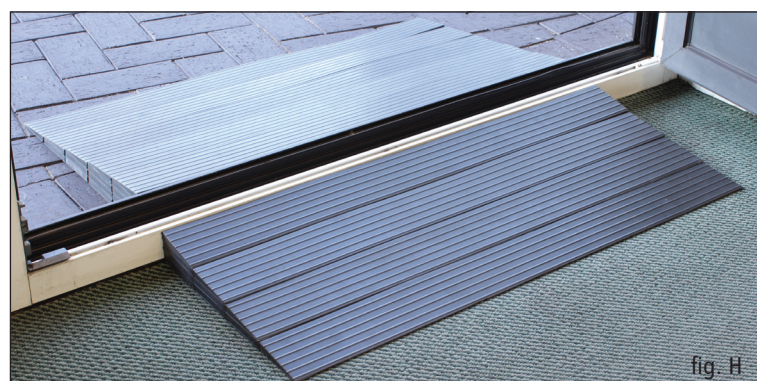
fig. H

6. OPTIONAL: To avoid the ramp from falling apart during transportation, the clip connectors VB1 and VB2 can be glued to the ramp parts. For the ramps with the article numbers -TS33, -TS44 and TS55 a suitable glue is already included in the scope of delivery (see fig. I + J).

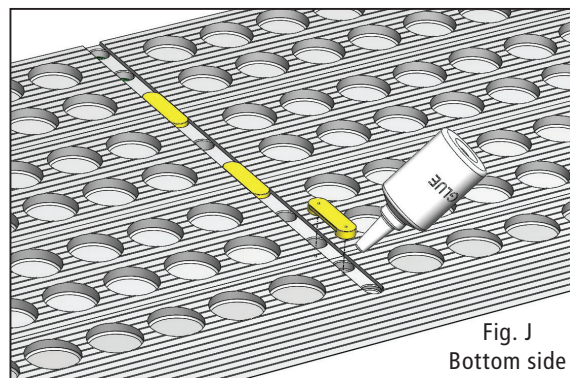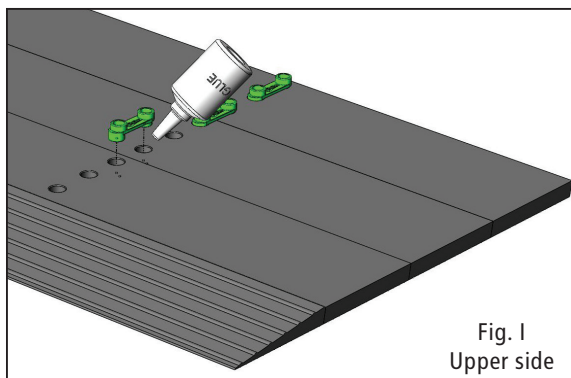05. Cleaning and Disinfecting

The threshold ramp can be cleaned with common household cleaners.

06. Maintenance and Service

Check the threshold ramp at suitable intervals but at least once a month. A visual inspection must be carried out. Check for external damage, such as broken pieces or cracks. Furthermore, it must be checked that the plug-in connection holds the individual rubber parts firmly together. Damaged door sill ramps must no longer be used and must be replaced immediately. If necessary, individual damaged elements can be purchased from a specialist dealer.

07. Re-use

The threshold ramp quick2go is suitable for reuse. It must be cleaned in accordance with section 05 „Cleaning and Disinfection“ and stored in a dry and UV-protected place.

08. Disposal

The threshold ramp quick2go can be disposed of with household waste. For this purpose, it must be disassembled into its individual parts to allow sorting of the materials.

09. Technical Data

Materials:	Rubber, PA6
Maximum load capacity:	250 kg
Width:	800 mm
Gradient:	approx. $7,8^\circ = 13,7\%$



Permissible environmental conditions:
Temperature range -20°C to $+50^\circ\text{C}$
Relative humidity maximum 80%

User Manual

Delivery Content

The delivery content varies according to ramp model:

Article number	Obstacle height	Number of ramp parts			
		Wedge	Block	VB1	VB2
Rampe-TS-11	11 mm	1	0	0	0
Rampe-TS-22	22 mm	2	1	2	2
Rampe-TS-33	33 mm	3	3	4	6
Rampe-TS-44	44 mm	4	6	6	12
Rampe-TS-55	55 mm	5	10	8	20
Rampe-TS-66	66 mm	6	15	10	30
Rampe-TS-77	77 mm	7	21	12	42

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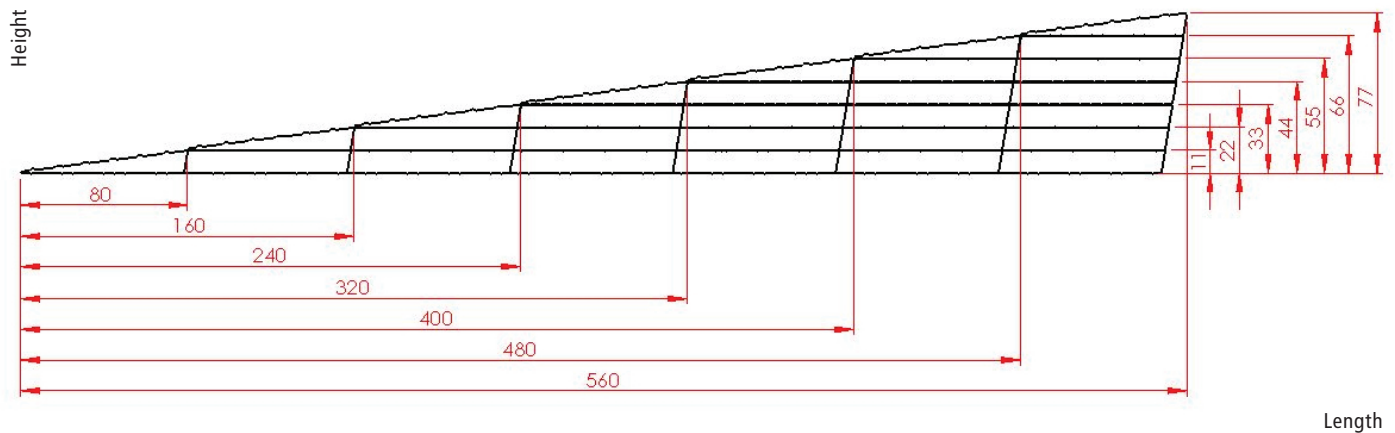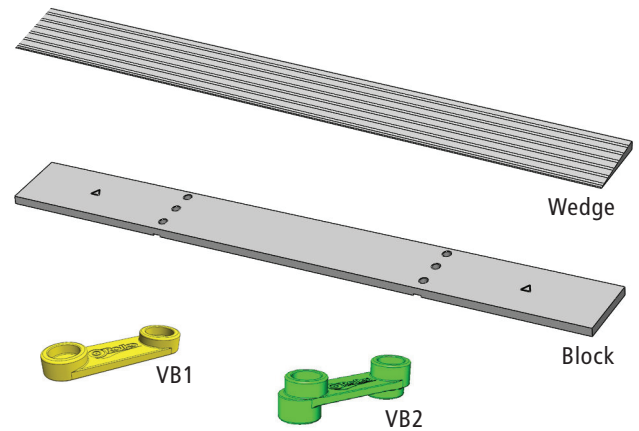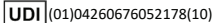
 Medical device

 Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. The EU declaration of conformity is available for download at instructions.rolko.com.

Operating Instructions: Door Threshold Ramp | quick2go / 2024-07-23 / Rev. 10

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen