JAY® Lite

IMPORTANT CONSUMER INFORMATION

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WARNING!

- Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

All Jay cushions are designed to help reduce the incidence of decubitus ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. Jay cushions are not a substitute for good skin care including; proper diet, cleanliness and regular weight transfers. In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue break down is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

Avoid sharp objects or exposure to excessive heat or open flame.

INTENDED USE

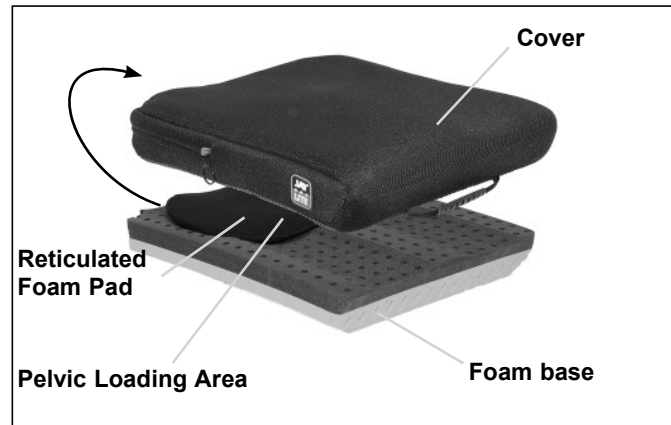
The Jay Lite is intended to provide the benefits of skin protection in a lightweight, comfortable and low-maintenance design. The Jay Lite incorporates a reticulated foam pad and an Olilex foam base with a microclimatic outer cover. The Jay Lite helps to provide pressure and shear reduction which may help reduce the incidence of decubitus ulcers. It is comfortable and doesn't restrict movement, allowing the user to change positions on the cushion.

The Jay Lite cushion can be used in any wheelchair, provided that there are the proper means of attachment (Velcro strips), as well as outside a wheelchair.

The intended lifetime of this product is 5 years.

The maximum user weight for this product is 150 kg.

JAY LITE COMPONENTS

FITTING A PRESSURE DISTRIBUTING CUSHION

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (2.5cm - 5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and pelvic loading area toward the rear. When fitted correctly your hips should be to the back of the chair. Your ischials (sitting bones) should be centered in the pelvic loading area of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your thighs rest firmly but comfortably on the cushion.

NOTE: Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the Jay Adjustable Solid Seat. Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

EASY MAINTENANCE & CLEANING

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.

Check the cover for tears and excessive wear and / or any other abnormalities.

Check the foam base to ensure foam consistency. Check the pelvic loading area insert to ensure reticulated foam consistency.

To clean cover

1. Remove the cover from the foam base and turn inside out.
2. Machine wash in warm water and drip or tumble dry on low heat.
3. No ironing required.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

To clean PLA (pelvic loading area) insert

1. Remove the insert from the foam base.
2. It is not necessary to remove the cover from the insert to wash it. We recommend it to be washed in the cover. Before washing, enclose the insert (with cover installed) in a mesh laundry bag to protect the Velcro hooks from the washing machine agitator and prevent tangling with other items.
3. Machine wash in warm water and drip or tumble dry on low heat.

Do not wash a damaged insert.

To clean foam base

1. Remove the cover from the foam base. Wipe lightly with a damp cloth.
2. Wipe off the base with a clean cloth. Let it completely air-dry before reassembly.
3. Reinstall the cover with the reticulated foam pad over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE:

- Consult table on next page for further approved cleaning methods

Laundry Care Symbols	
	Machine wash in hot water (70° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean
NOTE: 70°C is the maximum temperature. These instructions apply to the cover only.	

REASSEMBLING THE JAY LITE CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in "Easy Maintenance and Cleaning" then reassemble as follows:

Place the PLA insert on the foam base, fixing it to the base through the existing Velcro strips, and then introduce the base with PLA insert into the outer cover. Ensure that the zipper is at the rear of the foam base (the part number tag should be to the rear of the foam base) where the pelvic loading area is located.

HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray. To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly. Please be aware of the manufacturer's instructions for the disinfectant you are using. In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

CONDITIONS FOR SAFE STORAGE

1. Products should be stored in a dry place: at 20 – 75% Relative Humidity.
2. Protect from direct sunlight and dust for long periods.
3. Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
4. Position and stack the packaged products according to the "this side up" arrows on the packaging.
5. If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
6. Do not stack anything on top of the packaged products when the "do not stack" mark on the packaging is shown.
7. Product should never be stored outside and subject to the elements.

Approved Cleaning Procedures for JAY Cushions

Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
Outer Cover (Microclimatic and Stretch)									
(1). Wet Wash 70°C 									
Outer Cover (Incontinent)									
(2). Wet Wash 60°C 									
Inner Cover									
(3). Spray and Wipe disinfection  or Wet Wash 60°C	NA	NA	NA	NA	NA		NA	NA	
Cushion core (foam base)									
(4). Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
(5). Immersion disinfection 									
Cushion elements									
Air element	NA		NA	NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		

- 70°C is the maximum recommended temperature.
- 60°C is the maximum recommended temperature.
- The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.
- An autoclave is a self locking machine that sterilizes with steam under pressure.
- At low temperature (< 30°C)

The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute: http://www.rki.de/EN/Home/homepage_node.html

JAY LITE CUSHION WARRANTY

Every Jay Lite cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover. The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There is no implied warranty beyond what is contained herein. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

This does not affect your statutory rights.

Type:	Product Name/ SKU Number	XXXX-XX-XX	Date of Manufacture
Date:	LOT: defined by Julian Date	Part Code	Assembly Part Number
XXX mm	Seat width	MD	This symbol means Medical Device
XXX mm	Seat depth		Manufacturer's address
XXX kg	Max user weight	EC REP	European Authorised Representative
CE	CE Mark	UK RP	UK Responsible Person
UK CA	UKCA Mark	CH REP	Swiss Representative's address
	Consult instructions for use		Importer's address

ISO 7010-M002
Instruction manual/
booklet must be read!
(Blue Icon)

UK RP

Sunrise Medical Limited
Thorns Road, Brierley Hill
West Midlands, DY5 2LD
UNITED KINGDOM

SAMPLE

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

Indications

The varieties of cushions fitting variants, as well as the different positioning components to deliver higher positioning and skin protection benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning and skin protection needs and indications as follows[R7-R14] :

- Spinal Cord Injury
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Extremity defect deformity
- Amputation
- MS
- Joint contracture
- Tetraplegia

Contraindications

The cushion for a wheelchair shall not be used in case of [R3, R4]:

- Redness developed in the skin contact area
- Bottoming out present in the JAY Fluid Insert
- Decubitus

NOTE: General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S