
PROTEKTOR

Befestigungsset HA-SCH-BEF-90 für Rollatorschirm
Umbrella Base HA-SCH-BEF-90 for Rollator Umbrella
Kit de fixation HA-SCH-BEF-90 pour l'ombrelle-parapluie de déambulateur
Bevestigingsset BEF-90 voor rollatorparaplu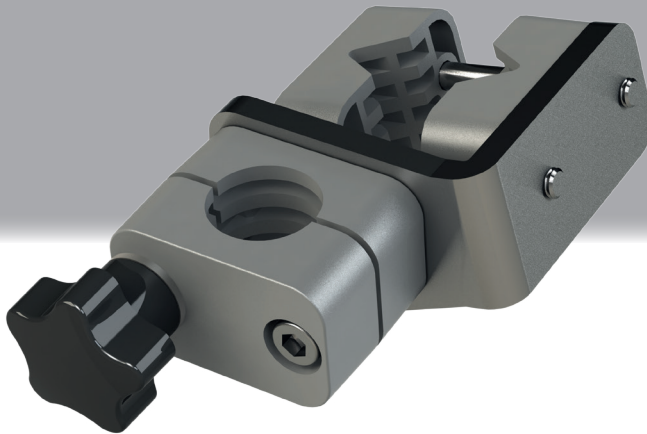

	Gebrauchsanweisung.....	Seite 02 - 03
	Operating instructions.....	Page 04 - 05
	Mode d'emploi.....	Page 06 - 07
	Gebruiksaanwijzing.....	Pagina 08 - 09

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Umbrella Base BEF-90 for PROTEKTOR Rollator Umbrella

Thank you very much for purchasing the umbrella-base and for your trust in ROLKO products. Before installing and using this product, please carefully read the entire user manual. Check that all parts are included and follow all of the safety instructions.

01. Application / Purpose

Using a umbrella-base BEF-90, a ROLKO PROTEKTOR can be attached to all common rollators and wheelchairs. The side-mount leaves enough space for the assembly of back-belts, cane holders, shopping bags, etc. The universal use of the BEF-90 allows a stable assembly to round-, oval- and custom-shape tubes with different inclination angle, at the right and left side of the rollator, or directly to the (horizontal) push handles.

02. Contraindication

The fastening unit BEF-90 is only to be used to attach the ROLKO PROTEKTOR rollator screen to the rollator or wheelchair. Compatibility cannot be guaranteed for umbrellas from other manufacturers or for attaching other accessories.


03. Safety Instructions

- With the BEF-90 bracket, only the Rolko Protector rollator umbrella may be attached to the rollator or wheelchair. The use of other umbrellas or accessories in connection with this holder is prohibited!
- Check firm connection of umbrella-base and umbrella before every use.
- The umbrella-base must be attached to the rollator in such a way that the brake function is not impaired, regardless, if the umbrella is open or closed. Also avoid any interference with lights, bell, cane holder, etc.
- Attach the umbrella-base in such a way to the rollator that the umbrella can be easily closed and folded away.
- Please note as well the safety instructions in the instructions for use of the Rolko PROTEKTOR rollator screen!
- The instructions for use are part of the BEF-90 mounting set. If the fastening set is passed on, the instructions for use must also be included.

04. Operation and attachment

The umbrella base is already prepared for oval tube attachment, at the right hand side of the rollator (fig. a+b). For other tube shapes, replace clamps and allen screws as needed.

- 1 Choose the matching clamps: for oval tubes (base setup, fig. a), for round tubes ($\varnothing$ 18/20/22/25 mm, fig. b), or mix-match clamps for custom tubes (fig. d).
- 2 Choose the appropriate Allen screw (35/45/60/75 mm, fig. f) and attach the umbrella base via Allen wrench in desired position to the height adjustable vertical handletube of the rollator.
- 3 Adjust the tilt angle of the umbrella base to the incline of the handle-tube, by first removing the Star screw completely. After adjustment, insert the Star screw back again into hole I or II of the bracket (fig. i+j).
- 4 For installation on the left hand side of the rollator turn the umbrella-base. Remove the Star screw completely and adjust the tilt angle of the umbrella base to the incline of the handle-tube. After adjustment, insert the Star screw back again into hole III or IV of the bracket (fig. g).
- 5 To attach the umbrella to a horizontal tube, first remove both, Allen screw and Star screw. Thereafter attach the umbrella base again to the bracket via holes I and IV (fig. e+h).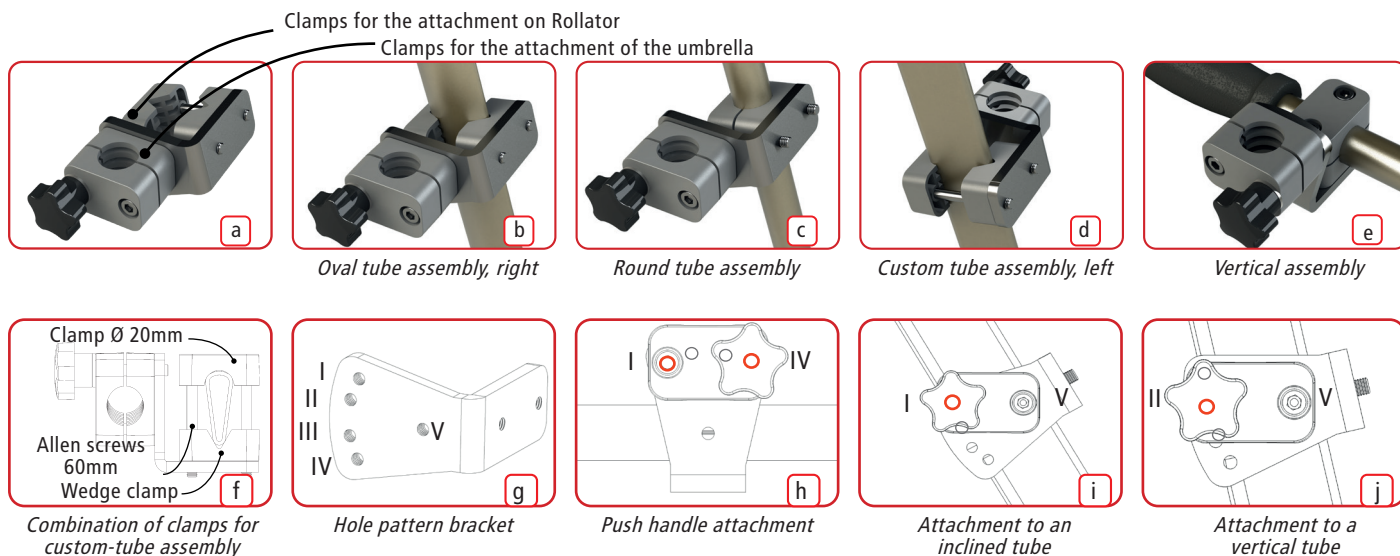

05. Cleaning and Desinfection

For cleaning, the fastening unit can be wiped with a damp cloth, if necessary with the addition of a mild detergent. Use lukewarm water and, if necessary, non-abrasive, pH-neutralized, commercially available household cleaners. If necessary, the fastening unit can be disinfected with household disinfectants by spray or wipe disinfection. Be sure to follow the manufacturer's instructions for use.

06. Maintenance

Check the fastening set regularly for firm seating and visual defects. Make sure that all individual parts used are undamaged.

07. Re-use

The product is suitable for reuse. The effects of weather and UV radiation, mechanical stress as well as natural aging or careless handling can cause premature wear, over which the manufacturer has no direct influence. In the case of defective or missing individual parts, this must be disposed of. For information on cleaning and disinfection, please refer to section 5.

08. Disposal

All parts of the mounting set are to be disposed of safely with household waste. To do this, disassemble the fastening set into its individual parts as far as possible to enable the materials to be sorted.

09. Technical Data

item no. HA-SCH-BEF-90

Weight: 0.53 kg

Suitable for pipes with Ø 18 - 25 mm and oval pipes up to 60 mm

Materials: steel, PA6

The HA-SCH-BEF-90 fastening set is available as a spare part for the following Rolko Protector rollator screens from specialist dealers:

- HA-SCH-1-BG(-REF)
- HA-SCH-1-GG(-REF)
- HA-SCH-1-R(-REF)


Delivery Content

1 Umbrella-base kit, pre-assembled, incl.:

- 2 clamps Ø 22 mm, for ROLKO PROTECTOR pickup
- 2 wedge-type clamps for oval tube assembly
- 2 Allen screws M6x60 / 1 Allen screw M6x35 / 1 Star screw M6

Accessories:

- Clamps for tube Ø 18/20/22/25 mm
- 2 Allen screws each M6x35/ M6x45/ M6x75
- Allen wrench
- Unser Manual

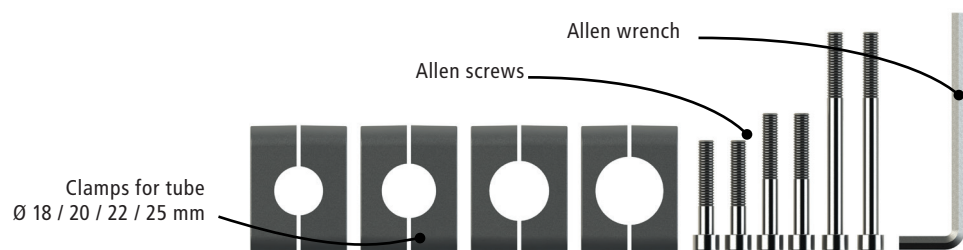Operating Instructions: HA-SCH-BEF-90 (Manual PROTEKTOR Base) / 2020-11-02 / Rev. 04

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Duitsland