quick2go® basic

Aluminiumrampen
Aluminium Ramps
Aluminium oprijplaten
Aluminiumramper
Rampes en aluminium
Rampe di alluminio

CE

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CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen

01. Purpose and Indication

The quick2go/basic ramps are used in order to overcome obstacles such as steps, stairs, kerbsides or to support the access into vehicles. The ramps are weatherproof and can be used indoors and outdoors. They are suitable for wheelchairs, power wheelchairs and other driving devices. The slip-resistant running surface allows a secure grip of the tyres and prevents a spinning of the wheels. The side walls of our ramps prevent a slip off of the wheelchair / driving device. These instructions are aimed at users of the ramps and support staff.

02. Contraindication

Before use, pay attention to the technical data in section 09, in particular to the maximum load capacity and the maximum obstacle height. Only wheelchairs and vehicles with 2 lanes are suitable for the quick2go and Rampe-2100 telescopic ramps. 3-wheel vehicles for the disabled cannot use these ramps.

03. Safety Instructions

- Please mind the operating instructions listed in this manual before using the ramps.
- Before use always check the safe hold of ramps on the obstacle surface. While using the ramps move the wheelchair or driving device only at a moderate speed. Sudden acceleration or braking need to be avoided.
- Always make sure that the ground and bearing surface are suitable and that the slip-resistant strip on the bottom part of the ramps rests on the entire bearing surface. The use in snow conditions, during frost or rain may occur only with the utmost care.
- When the ramps are used in combination with a vehicle, this must be secured against rolling away.
- For safety reasons we recommend to not operate the ramps alone, but always with a supportive person, especially when carrying, unfolding or extending the ramps.
- We recommend wearing gloves to carry and put on the ramps.
- Do not disassemble the ramps.
- Note the information regarding the maximum load capacity, tread width and obstacle height. Never exceed the specified maximum values listed in this manual.
- Foldable ramp (K1800): caution - danger of crushing! When unfolding and folding the ramp there is a risk of crushing the fingers.
- Telescopic ramps: after extending or sliding the ramp parts together, always insert the safety pin and turn it to lock the ramps. To avoid an unintended extension of the ramps NEVER hold them vertically while unlocking the safety pin or extending the ramp. An uncontrolled extension threatens with injuries to the legs and feet. During use, both ramps must be parallel to each other aligned and laid out at the same height, the distance between ramps must correspond to the track width of the wheelchair or vehicle.
- The instructions for use are an integral part of the ramp and must be passed on if the ramp is passed on.
- If serious incidents occur in connection with this product, the incidents must be reported to Rolko Kohlgrüber GmbH and to the competent authority of the EU Member State in which the user or patient is based.

04. Operation/ Use

Before using the ramps for the first time please check those for any damages.


Loading Ramps: RAMPE-A600, RAMPE-A900, RAMPE-A1200, RAMPE-A1500


Transport the ramp preferably with a second person and always hold the ramp on the side profiles

Lay the ramp with the slip-resistant strip downwards onto a firm obstacle surface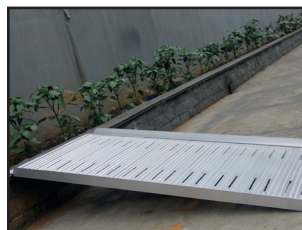

Make sure that the loading ramp lays properly on the obstacle surface

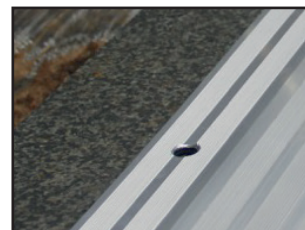Optional: attach the ramp to the obstacle surface (fixation pin not included)

Foldable Ramp: RAMPE-K1800

While carrying the ramp hold it by the handle. Never underestimate the weight and length of the ramp!

Unfold the ramp completely and lay it with the slip-resistant strip downwards onto a firm obstacle surface

Make sure that the foldable ramp lays properly on the support surface


Optional: attach the ramp to the obstacle surface (fixation pin not included)


Instruction Manual

Telescopic Ramp: RAMPE-2100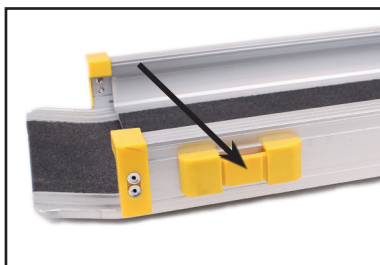


Hold the ramp horizontally, then press and hold the safety button on the ramp end

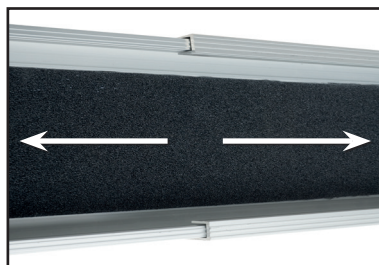
Fully extend the ramp horizontally until it locks a "click" sound

Lay the ramp with the slip-resistant strip downwards onto a firm obstacle surface

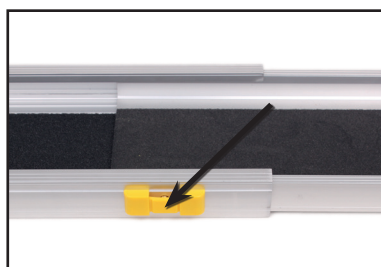
After the use: hold the ramp horizontally and press the safety button in the middle of the ramp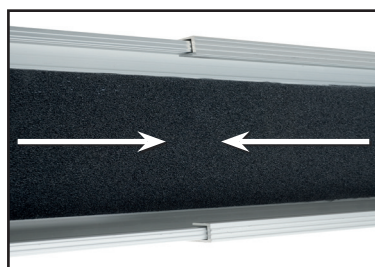


Keep holding the ramp horizontally and slide both ramp parts together until the ramp locks a "click" sound


Telescopic Ramps: RAMPE-T1, RAMPE-T2, RAMPE-T2/3, RAMPE-T3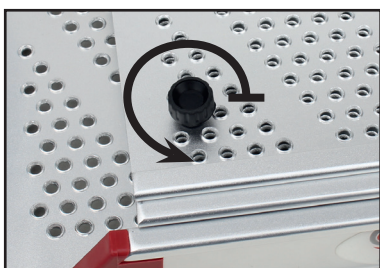

Remove the transport protection screw on the running surface before the first use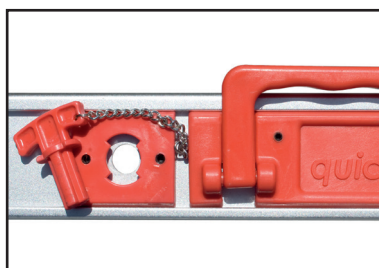

Hold the ramp horizontally by the handle, turn the safety pin and then pull it out

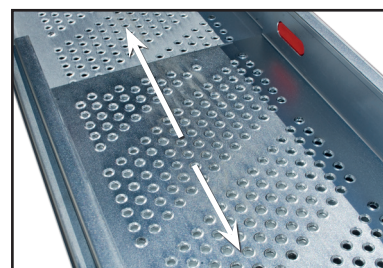Completely pull the movable part(s) of the ramp horizontally

Lay the ramp with the slip-resistant strip downwards onto a firm obstacle surface

The ramp may drive at low speed and without accelerating.

After the use: slide the ramp parts together, insert the safety pin and turn it to lock the ramp

05. Cleaning and Desinfection

Wash off the ramps with a damp cloth, sponge or brush and then let them dry completely. Do not use chemicals. Before every use ensure that specially the running surface, moving parts, slip-resistant strip of the ramps are free of dirt, grease, oil and other impurities, such as leaves or sand. A ramp disinfection is not necessary under normal use. If disinfection is required in exceptional cases or for reuse, use a mild, commercially available disinfectant. Follow the manufacturer's instructions. Disinfectants containing phenol or phenol derivatives, aromatic hydrocarbons, organic or inorganic substances with active chlorine, formaldehyde or other aldehydes, and bases are not permitted.

06. Maintenance

There is no limitation of service life for the ramps quick2go/basic. The ramps are thus reusable, provided a careful and periodic inspection is performed. Inspect the ramps before each use for any damages (running surface, aluminum profiles, plastic parts, slip-resistant strip, etc.) and specially the proper function of the moving parts (safety pin/button, screw joints, rivet joints, hinges etc.). An improper use of the ramps may cause damages or breakage. Defective or damaged ramps and also ramps with strong signs of wearout are not allowed to be used and need to be replaced immediately.

07. Re-use

The product is suitable for reuse. The effects of weather and UV radiation, mechanical stress as well as natural aging or careless handling can cause premature wear, over which the manufacturer has no direct influence. See section 06 Maintenance. Clean and disinfect the ramp according to section 05: Cleaning and disinfection.

08. Disposal

Disassemble the ramp as much as possible. Small and plastic parts can be disposed of with household waste. Larger aluminum parts and profiles are accepted by recycling.

09. Technical Data

Loading Ramps

Material: aluminium, rubber
Modell: non-foldable, 1-part, 60 cm, RAMPE-A600
non-foldable, 1-part, 90 cm, RAMPE-A900
non-foldable, 1-part, 120 cm, RAMPE-A1200
non-foldable, 1-part, 150 cm, RAMPE-A1500

Item no.	Weight (kg)	Load capacity* (kg)	Length (cm)	Width (cm)	Max. obstacle height (cm)** 	Max. obstacle height (cm)*** 	Max. tread width (cm)	Sidewall height (cm)
RAMPE-A600	4,5	300	60	75,6	12	3,6	75	3,5
RAMPE-A900	6,7	300	90	75,6	18	5,4	75	3,5
RAMPE-A1200	9,2	300	120	75,6	24	7,2	75	3,5
RAMPE-A1500	13	300	150	75,6	30	9	75	3,5

*The max. load capacity applies to a tyre width of at least 10 cm and with an even load, distributed on the drivable ramp surface. **For powered wheelchairs or use with a helper. ***For self-drivers.

Foldable Ramp

Material: aluminium, rubber
Modell: foldable, 1-part, 182,4 cm, RAMPE-K1800

Item no.	Weight (kg)	Load capacity* (kg)	Length (cm)	Width, unfolded (cm)	Max. obstacle height (cm)** 	Max. obstacle height (cm)*** 	Max. tread width (cm)	Sidewall height (cm)
RAMPE-K1800	14	300	182,4	71,4	36	11	69	3,5

*The max. load capacity applies to a tyre width of at least 10 cm and with an even load, distributed on the drivable ramp surface. **For powered wheelchairs or use with a helper. ***For self-drivers.

Instruction Manual


Telescopic Ramps

Material: aluminium, PA6 GF15, PA6 GF30, PP GF20, POM, rubber, ABS
Modell: non-telescopic, 1-part, 110 - 110 cm, RAMPE-T1
telescopic, 2-part, 121 - 210 cm, RAMPE-2100
telescopic, 2-part, 110 - 200 cm, RAMPE-T2
telescopic, 3-part compact, 87 - 200 cm, RAMPE-T2/3
telescopic, 3-part, 110 - 285 cm, RAMPE-T3

Item no.	Weight (kg)		Load capacity*(kg)		Min. - max. length (cm)	Width (cm)	Max. obstacle height (cm)** 	Max. obstacle height (cm)*** 	Max. tread width (cm)	Sidewall height (cm)
	Piece	Pair	Piece	Pair						
RAMPE-T1	3,4	6,8	150	300	110 - 110	23,72	22	7	21	5
RAMPE-2100	4,4	8,8	135	270	121 - 210	19,40	42	13	14,5	4,4
RAMPE-T2	6,5	13	150	300	110 - 200	23,72	40	12	19,5	5
RAMPE-T2/3	6,8	13,6	130	260	87 - 200	23,72	40	12	18	5
RAMPE-T3	9,3	18,6	130	260	110 - 285	23,72	57	17	18	5

*Max. load capacity when an even load is applied on the drivable surface of both ramps. **Max. obstacle height for powered wheelchairs or use with a helper. ***Max. obstacle height for self-drivers.

Label

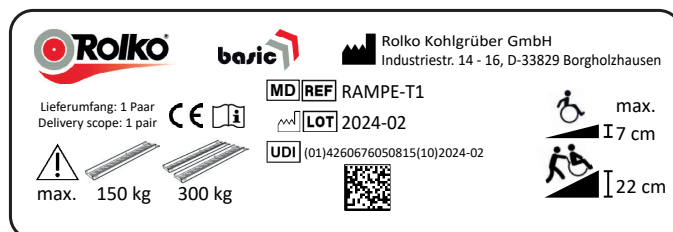Example: RAMPE-T1, symbol explanation on page 27.

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. This manual and the EC Declaration of Conformity for the product in PDF format are available at www.rolko.com. Also, make use of your browser's screen reading function.

Operating Instructions: Aluminium Ramps quick2go / basic / 2024-02-16 / Rev. 21

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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CH-REP. CH-Importeur: INNOVAhelf GmbH, Thunstr. 5a, CH-3110 Münsingen

01. Intended purpose, area of use

The ramps are used to move over obstacles such as single steps, thresholds and curbs. They are suitable for manual wheelchairs, electric wheelchairs, and potentially for other vehicles. The slip-resistant surface of the ramps ensures a firm grip for tires, and prevents driven wheels from skidding.

02. Contraindication

The ramps are not suitable for any application that deviates from the intended use and that could result in hazards. This includes exceeding the maximum load capacity (see table) or an uneven distribution of the vehicle weight on the ramp. The wheel spacing of the wheelchair may not exceed 650 mm.

Each ramp is suitable only for bridging specific height differences - see section 10: Technical data.

03. Safety information

- Before using the ramp, ensure that it sits firmly on the contact surface. Users may only move onto the ramps at a moderate speed. Avoid sudden accelerating or braking.
- Always ensure that the subsurface is appropriate, and that the slip-resistant strips on the bottom of the ramps are in full contact with the subsurface. Take the greatest possible caution when using the ramps in the snow, frost, or in wet weather.
- For safety reasons, we recommend that you never use the ramps alone; instead, always use them with the help of a support person.
- We recommend wearing gloves when carrying and placing the ramps.
- Observe the information provided on load-bearing capacity, wheel spacing, and obstacle height. Never exceed the maximum values.
- This instruction manual is part of the ramp and must be passed on to any new users together with the ramp.
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH and the authority responsible in the EU member state in which the user is based.

04. Description and handling

Threshold ramps

The threshold ramps consist of a reinforced and welded aluminium checker plate panel. The ramps are placed in front of the step or threshold to be crossed, and can remain in place there. They can be used on a mobile basis. Threshold ramps are suitable for indoor and outdoor use. They are available for thresholds at heights of 60, 80 and 100 mm. The max. load-bearing capacity is 300 kg.


Threshold bridge

The threshold bridges are made of a bent, reinforced sheet of aluminium with a slip-resistant surface. They are placed over the threshold to be crossed, and can remain in place there. They can be used on a mobile basis. Threshold bridges are suitable for indoor and outdoor use. They are available for thresholds with maximum dimensions of 26x130 and 75x130 mm (height x width). The maximum load-bearing capacity is 300 kg.

05. Ramp structure


Threshold ramps

The tread surface (1) is made of slip-resistant checker plate. The rubber coating (2) serves as the slip-resistant contact surface of the ramp. The tread surface is reinforced on the bottom (3).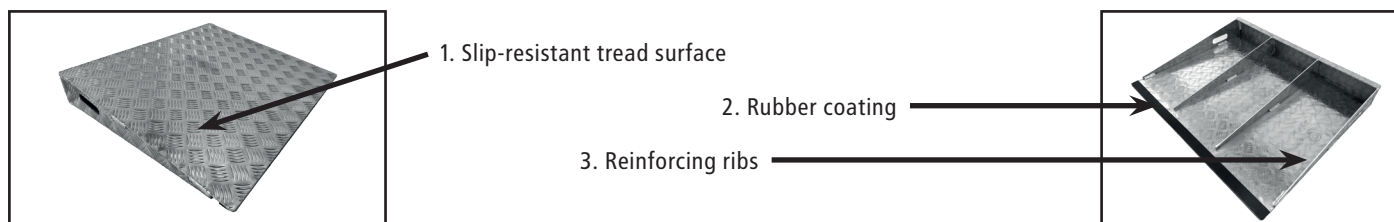

Threshold bridge

A slip-resistant material (2) is adhered to the bent aluminium sheet (1). The screw-fitted brackets (3) form an additional contact surface, and determine the maximum height and width of the threshold.

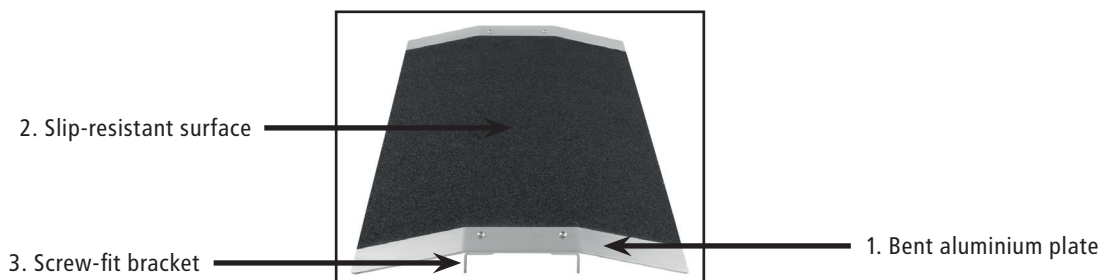Instruction Manual

06. Cleaning and disinfection

Wash the ramps with a moist cloth, sponge, or brush, and then dry. Do not use chemicals. Before each use, ensure that the tread surface and slip-resistant strips of the ramps are free from dirt, grease, oil, and other contaminants. It is not necessary to disinfect the ramps during normal use. If disinfection is necessary in exceptional cases, or for reuse, please use a commonly available, mild disinfectant. Observe the manufacturer's usage instructions. Disinfectants containing phenol or phenol derivatives, aromatic hydrocarbons, organic or non-organic substances containing active chlorine, formaldehyde, or other aldehydes and alkaline solutions are not permitted.

07. Maintenance

The ramps do not have a restricted service life. The ramps are suitable for reuse, as long as they are inspected regularly and thoroughly. Before each use, inspect the ramps for damage (tread surface, slip-resistant elements, welded connections, etc.). Improper use of the ramps can result in damage or breakage. Defective or damaged ramps, as well as ramps with signs of wear, should not be used any longer and must be replaced immediately.

08. Reuse

The product is suitable for reuse. The influences of weather and UV rays, mechanical loads, natural ageing or improper handling can cause premature wear. The manufacturer has no direct influence over such wear and tear. Observe section 06, Maintenance. clean and disinfect the ramp according to section 06.

09. Disposal

The ramp is made primarily of aluminium. It is accepted by recycling agencies.


10. Technical data

Material: Aluminium, EVA

Versions:

Item no.	Weight (kg)	Load-bearing capacity* (kg)	Length (cm)	Width (cm)	Max. obstacle height (cm)** 	Max. obstacle height (cm)*** 	Max. wheel spacing (cm)
RAMPE-AW-TR301-40	3.2	300	40	72	6.0	-	65
RAMPE-AW-TR301-50	4.2	300	50	72	8.0	-	65
RAMPE-AW-TR301-60	5.2	300	60	72	10.0	-	65
RAMPE-BT-30-400	3.1	300	40	70	2.6	13	63
RAMPE-BT-80-900	7.2	300	90	70	7.2	13	63

*Max. Load-bearing capacity applies for a tire width of at least 10 cm and with the ramp surface evenly loaded. **Electric drive or use with support person. ***For self-propelled users. The ramp may not be used for higher or wider obstacles.

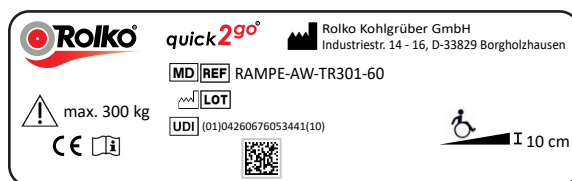Product label - example: RAMP-AW-TR301-60, Explanation of symbols provided at the end of this user manual.

As an accessory for a class 1 medical product, the product carries the CE label in accordance with Regulation (EU) 2017/745 (MDR) and thus meets all German and European safety requirements applicable to the product. The product meets the relevant sections of the DIN EN ISO 21856 standard. This instruction manual and the associated EC Declaration of conformity are available for download at rolko.com.

Instruction manual: Threshold ramps and bridges quick2go / 2025-11-20 / Rev. 3
Basic UDI DI: 426067605RAMPE00000000ZD

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Reproduction, in whole or in part, only with prior permission of the publisher.

All rights reserved. The delivered product may differ from the images in this manual.

All technical data is nominal data and subject to tolerances due to construction and manufacture.

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