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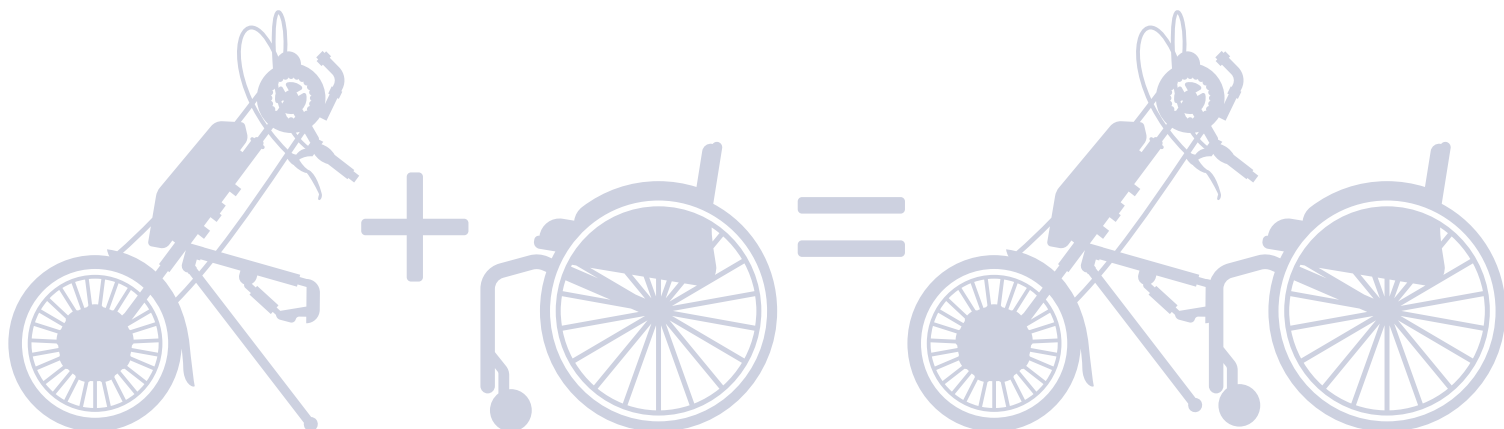
Operating guide: GX display: GX-Display

This operating guide provides you with an overview of the most important features and functions of the GX display. For detailed information and instructions, please refer to the manual supplied with your adaptation device.

Switching on/off

Switch on the key switch and the battery. Press the „“ button, which is located on the underside of your display, for approx. 2 seconds.


Changing the display mode of the display

To switch between the views, quickly press the „“ button located on the underside of your display.

Information and settings for the trip

Level 1 - 5: The power assist device can be operated in 5 different levels. These differ in terms of maximum speed. In level 1, the power assist device moves at the slowest speed. In level 5, it travels at maximum speed. Acceleration is not possible at level 0.

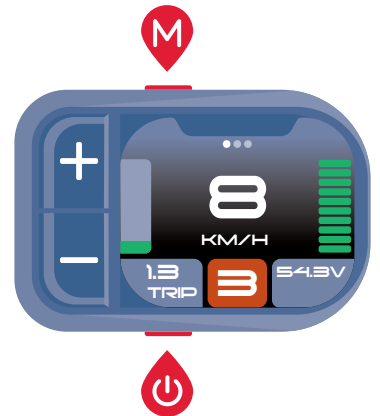Trip: Press the „“ button on the top of your display to change the information displayed, such as „Trip“ for driving.

Starting off on inclines

1. Hold the brake without brake cut-off switch (a brake with brake cut-off switch deactivates the motor as long as it is actuated). In most equipment, this is the left brake lever. However, this varies de-

pending on the type of equipment and individual adaptations.

2. Accelerate with the throttle until the power assist device wants to move for-

ward

3. Release the brake slowly until you are moving forwards. Accelerate faster if you are rolling backwards..

Walk mode

Use case

Walk mode is useful if you are accompanying a pedestrian.

It ensures that your power assist device accelerates to a speed of 3 km/h and maintains this speed as long as you keep the corresponding button pressed.

Switching on

Press the „“ button located on the front of the display. Hold the button down for as long as you wish to use the walk mode.

Switching off

Walk mode switches itself off as soon as you release the button.